

# CUSTOM DOLOMITE ROLLATORS

The Dolomite Alpha models are our rehab rollators with forearm pads for more stability and upright posture when walking or gait training. But the Alpha comes in one size and doesn't fit all. We get many inquiries for a rehab rollator with a greater capacity or to accommodate shorter persons. Clarke Health Care is now customizing other Dolomite models to fit more body dimensions.

While the Alpha still has the greatest arm trough height range, sometimes there is need for a shorter seat or height. We can use the Legacy or Symphony Low or Super Low models. When a capacity greater than 275 lb. is needed, we can customize the Maxi with its 452 lb. capacity and a wider frame. Note: each of these customizations will have limitations on handle height that can be achieved.

Options for customizing also include alternate braking, such as one hand brakes mounted on either side or on a center bar or as Hemiplegic handles with one hand brakes. A full pad can be mounted instead of individual troughs. It permits the user to lean in at any point and reach across for the brake handles.

We use the Alpha style brake handles with the forearm pads, Advanced is a joystick style handle which can be rotated for best position with a push/pull motion. Basic is the looped brake handle turned vertically for a squeeze/push brake action. Either of these styles can be mounted as a one-hand system.


We quote each custom build. Use the worksheet to indicate the client's needs. Once the quote is approved the custom rollator is assembled in our shop. We do not fabricate any new parts, rather we use existing parts from various Dolomite models to meet your needs. Allow several days for us to build a custom rollator.



Maxi with Advanced brakes, forearm pads and slow downs



Maxi Low with Basic brakes, forearm pads

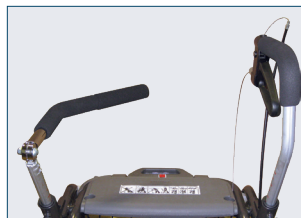

HP Left on Symphony



Maxi+ standard with forearm pads, Basic brakes



Symphony Low with full platform, Basic brakes



Legacy with Cross bar left



Legacy with one pad and grip only, one hand brake mounted left

**CLARKE HEALTH CARE PRODUCTS, INC.**


7830 Steubenville Pike • Oakdale, PA 15071

phone (888) 347-4537 • Fax (724) 695-2922 • [www.clarkehealthcare.com](http://www.clarkehealthcare.com)

## Specifications

MODEL	CAPACITY	HANDLE HEIGHT	WITH PADS, ADDS 3.5"	WITH PLATFORM, ADDS 4"	WITH TALL HANDLE	TALL HANDLE WITH PADS	TALL HANDLE WITH PLATFORM
Legacy D12050B	330 lb.	33"-37"	36.5"-39.5"	37"-41"	36"-40"	40.75"-43.75"	39"-43"
Legacy Low D12052	330 lb.	31"-34"	35.5"-38.8"	35"-38"	34"-37"	38.75"-41.75"	38"-41"
Legacy SuperLow D12054	330 lb.	28"-31"	32.5"-35.5"	32"-35"	31"-34"	35.5"-38.5"	35"-38"
Symphony D12160	275 lb.	30"-36"	33"-39"	34"-40"	NA	NA	NA
Symphony D12161	275 lb.	27.5"-33"	30.5"-36"	31.5"-37"	NA	NA	NA
Symphony D12162	275 lb.	25"-30"	28"-33"	29"-34"	NA	NA	NA
Maxi D12120	452 lb.	32"-38"	35"-41"	36"-42"	36"-42"	39"-45"	40"-46"
Maxi Low D12122	452 lb.	28.5"-35"	31.5"-38"	32.5"-39"	32.5"-39"	35.5"-42"	36.5"-43"
Alpha D12220 or D12230	275 lb.	37.5"-47"	included	38.5"-48"	NA	NA	NA
FuturaTall D12076T	330 lb.	37.75"-43.5"	40.75"-46"	41.75"-47.5"	NA	NA	NA

### ARM POSITIONING OPTIONS

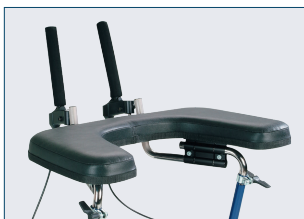

Full platform, 27" W x 15.5" D. Showing advance brake handles. Adds 4" to handle height. Choose brake style.




Forearm trough/pad. Adds 3" to handle height. Padded cradle supports forearm with brake handles mounted forward. Choose brake handle style.




Tall handle extensions. Replaces handles tubes on Legacy or Maxi+. Legacy tall handles will add 3" handle height. Maxi tall handles will add 4" handle height. Can be mounted with foam grip or anatomic grip.

*Restrictions: The custom walker will be built in our shop. It cannot be shipped totally assembled. The handles will need to be inserted and adjusted for height and length. We will have brake cables attached and adjusted. Customs or Alphas are not recommended for drop ship unless a technician can be present to make adjustments to fit the client. Arm troughs may block the use of the seat. The full platform will block the seat. Parts indicated on the worksheet do not indicate all components necessary to complete the walker assembly. Your custom quote will show all necessary pieces.*


### BRAKE OPTIONS

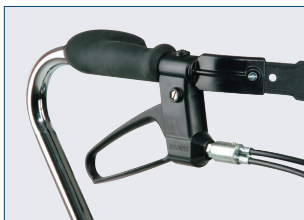

One hand brake for Legacy or Maxi+. 2 cables from one side operate brakes at both wheels. Used without pads. Symphony also has one hand brake with different handle.

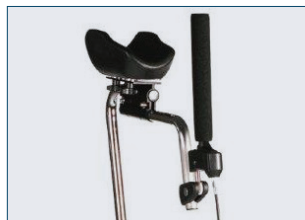

Advanced brake, joystick style used with arm pad or full platform. Can be mounted as one hand or two hand braking. Push/pull action. Handle can be rotated in clamp to change push direction.

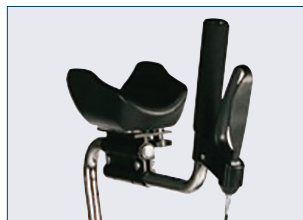


Basic brake handle, looped brake handle turned vertical. Can be mounted for two hand or one hand braking. It is push/squeeze action.




Basic one hand brake, 2 cables from one side will operate brakes at both wheels.


### ALTERNATE BRAKE OPTIONS

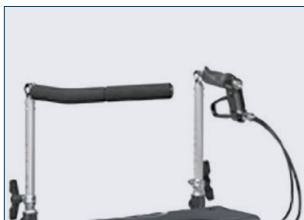

HP Handle, Hemiplegic handle right, with left one hand brake. Foam covered tube allows pushing with forearm or chest. Replaces backstrap. Can only be mounted on Symphony models.

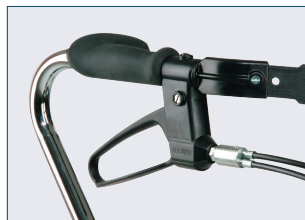

One hand brakes are available on Legacy, Symphony, Maxi, Alpha and Futura Tall. 2 cables come from one side to operate brakes at both wheels.

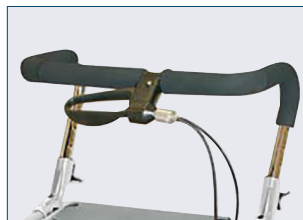

Crossbar, replaces backstrap. One hand brake handle is offset mounted in center. Will not fit Maxi+.



Slow down brakes mount on rear wheels. They are independent of the braking system. A nylon wheel rides on the tire to add tension to slow the walker. Knob on top adjusts the tension on each side.

## CLARKE HEALTH CARE PRODUCTS, INC.

7830 Steubenville Pike • Oakdale, PA 15071

phone (888) 347-4537 • Fax (724) 695-2922 • [www.clarkehealthcare.com](http://www.clarkehealthcare.com)