DOLOMITE *ALPHA ADVANCED*

SV BRUKSANVISNING

NO BRUKSANVISNING

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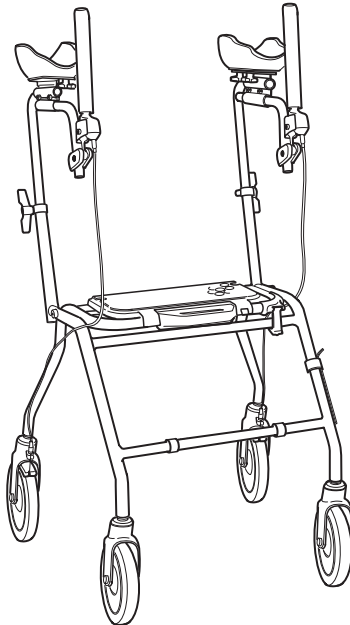
GB INSTRUCTIONS

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
DOLOMITE ALPHA ADVANCED

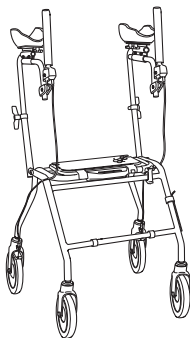SV BRUKSANVISNING

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GB OPERATING INSTRUCTIONS

Study the operating instructions carefully **before** use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

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GB ADJUSTMENT POSSIBILITIES

• Height adjustment	8
• Inwards rotation of arm supports	8
• Depth adjustment	10
• Inwards rotation of arm support plates.....	10
• Angular setting of handle	10
• Depth adjustment of the arm support plates	10
• Adjustment of distance between arm supports.....	8,12
• Inwards rotation of arm support plates	12
• Height adjustment of brake.....	14
• Rotation of brake handle.....	14

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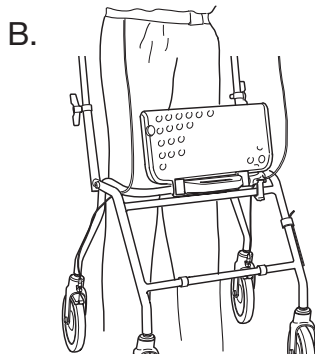
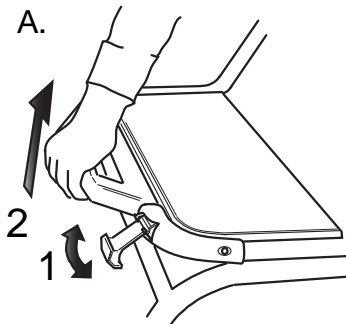
GB FOLDING/SEAT

- A. Pull the folding blocker and lift the handle for folding. Folded position. Space-saving. Suitable for carrying in stairs. For ready-to-start position, block with thumb for locking. A clicking sound indicates right position. Make sure that the folding blocker is in place when the walker is to be used again.
- B. The seat can be folded to give more walking space.

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GB ADJUSTMENT OF BRAKE HANDLE

1. Loosen the screw (1) with the attached key, Important! Make sure the socket head wrench is fully fitted before starting to tighten or slacken.
2. The fastening device (2) can now be height and sideways-adjusted and the brake can by means of the ball joint be rotated for optional hand location and optional function direction.
3. Tighten the screw (1) again after the brake has been set as desired.
4. The walker is stopped at parking by moving the handle in the opposite direction.

A clear clicking sound indicates that the brake is in the right position. It is important that both brakes are set in the parking position. To release the brake - pull backwards.

Note: The brake can be set at a greater angle by twisting the pipe (3) in connection with the depth adjustment

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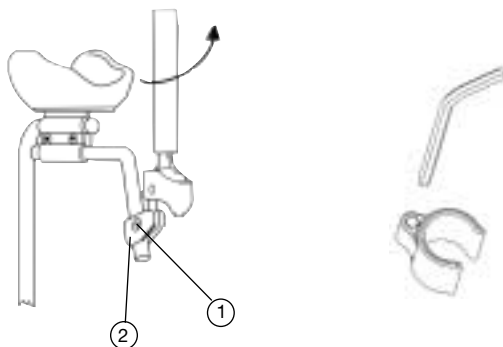
GB BRAKES

Adjust the brake wire using cap keys Nos. 8 and 10 in case of poor brake effect.

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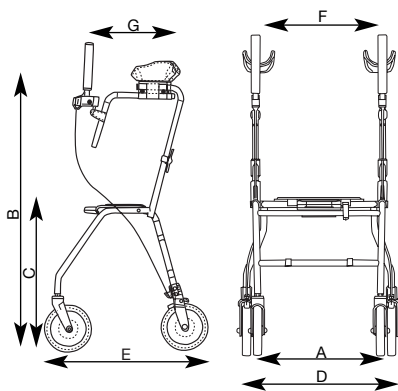
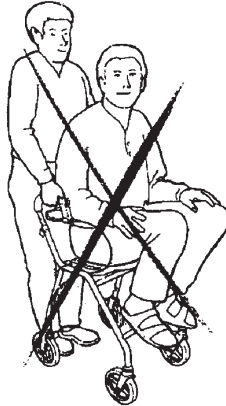
GB It is **prohibited** to stand on the walker and to use it for transport of people.

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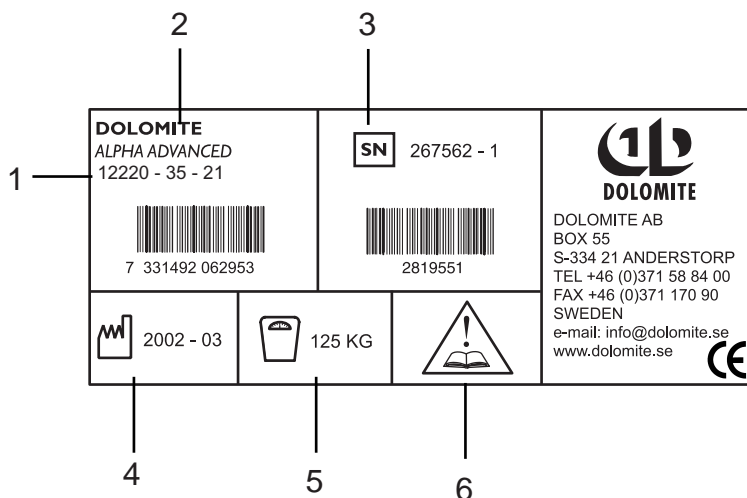
GB All technical data available on the **DOLOMITE ALPHA ADVANCED** label:

- 1: Article number
- 2: Name of product
- 3: Serial number
- 4: Date of manufacture
- 5: Maximum user's weight
- 6: Attention: see instruction for use

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GB**INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:**

When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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GB MAINTENANCE - SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments** are properly tightened.
- Lubricate the wheels regularly with grease.
- **Immediately contact the nearest dealer if a fault should appear. Never use a defective walker.** Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid! User's max. weight: 275 lbs. (125 kg, 20 stones.) DOLOMITES will not accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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DOLOMITE AB
AQUATEC GmbH & Co.KG
MOBITEC Mobilitätshilfen GmbH