

Arcorail Assembly Instructions for REGULAR HOME STYLE beds (AR25)

FITS HOME STYLE METAL BED FRAMES
(1" - 1 3/4") IN HEIGHT




Regular Home Style beds have
ANGLE IRON FRAMES that are
horizontal and flat at the bottom.

Note: If your Home Style bed does not have a metal frame then use our M-Rail model.
www.hartmobility.com


Please read the following instructions before you start to assemble the Arcorail bedside handrail.
This document contains important safety information.

Box Contents


Item	Qty.	Description
A	1	HANDLE
B	1	HANDLE ATTACHMENT
C	1	BACK ANCHOR ASSEMBLY
D	1	FRONT ANCHOR TUBE ASSEMBLY
E	1	LONG POST
F	1	MEDIUM POST
G	1	SHORT POST
H	2	EXTENSION TUBE
I	1	HEIGHT ADJUSTING PIN
J	1	POST CAP
L	1	3/16 ALLEN KEY
M	1	5/32 ALLEN KEY
N	1	LARGE HOME STYLE CLAMP
O	1	SMALL HOME STYLE CLAMP

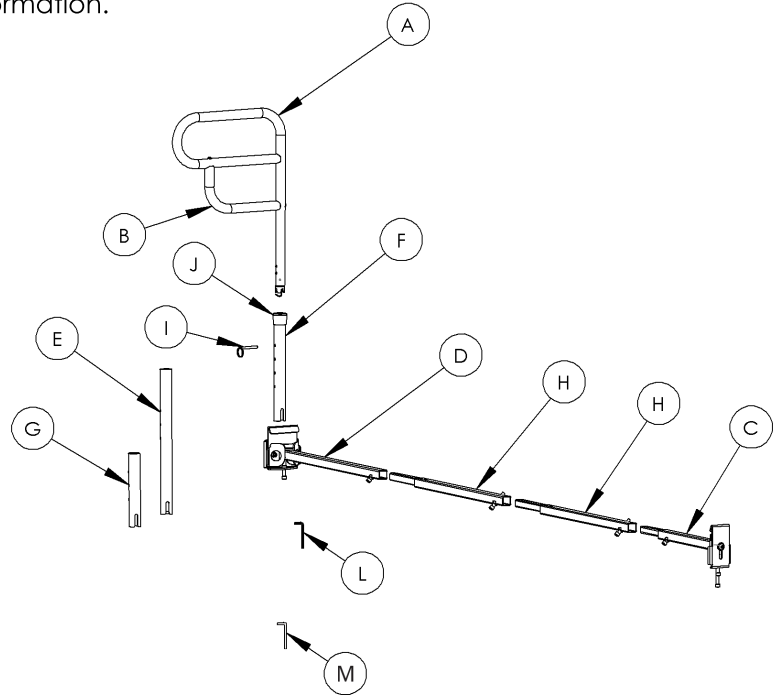

NOTE: ITEMS (N) AND (O) ARE PRE-INSTALLED

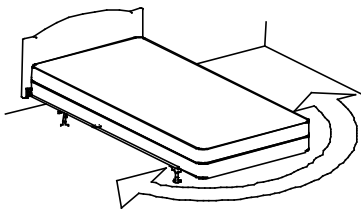


If necessary, move the bed away from the wall so that both sides of the bed are easily accessible.

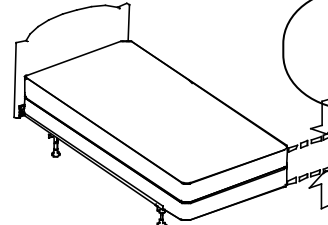


Measure and record the total thickness of the mattress and mattress base/box spring.

©2005 Hart Mobility, a division of Andrew J. Hart Enterprises Inc.
All rights reserved. Information in this document is subject to change without notice. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Hart Mobility. Other products and company names herein may be the trademarks of their respective owners.
272AR25 Rev. 1 14/01/07

Pick Post Length & Handle Configuration

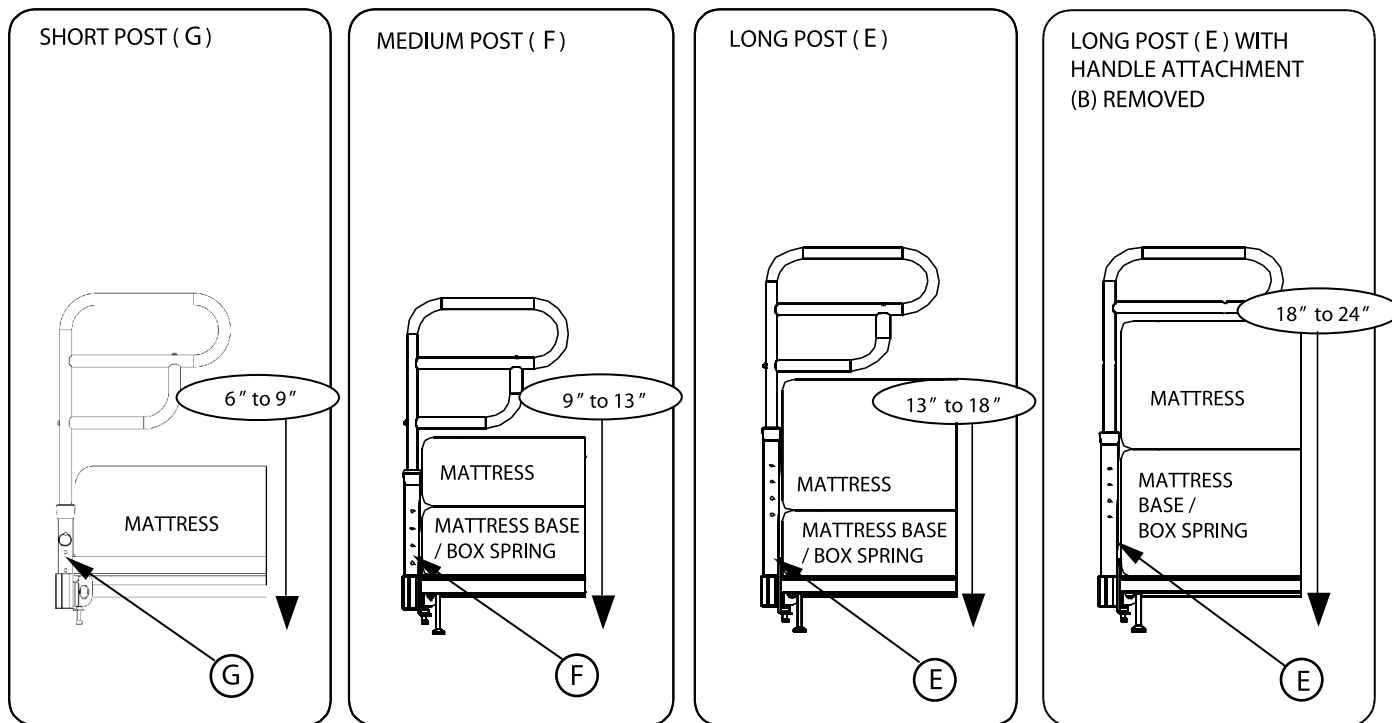

Pick Frame Width

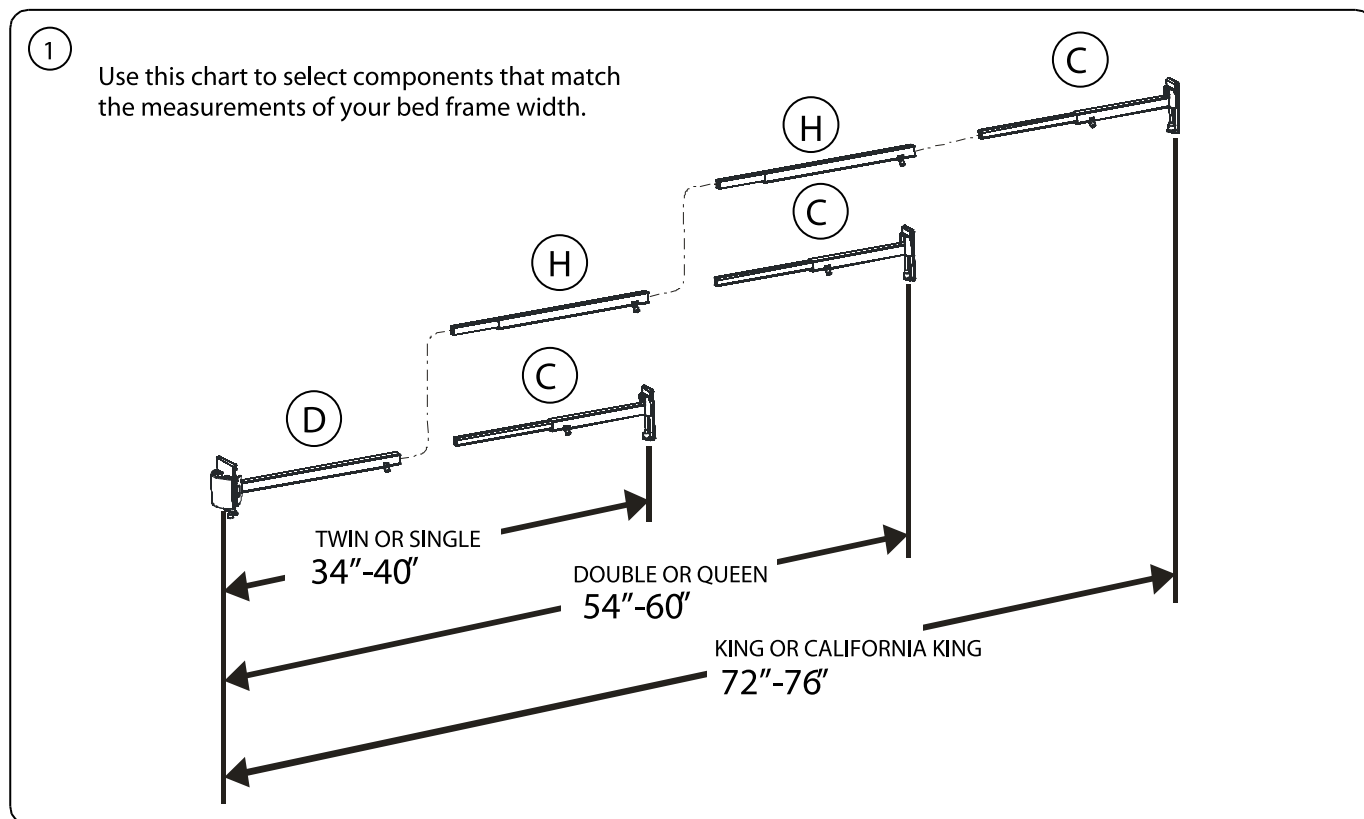


©2005 Hart Mobility, a division of Andrew J. Hart Enterprises Inc.

All rights reserved. Information in this document is subject to change without notice. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Hart Mobility. Other products and company names herein may be the trademarks of their respective owners.

272AR25 Rev. 1 14/01/07

2

All moving parts must be pre-fastened snugly and easily adjusted by hand.

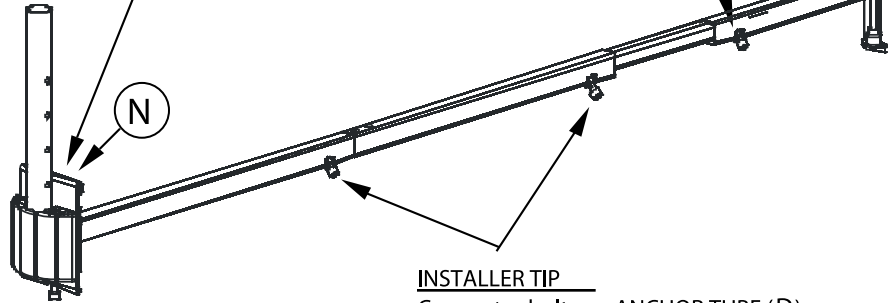


INSTALLER TIP

Connector bolts on ANCHOR TUBE (D) and CONNECTOR TUBES (H) (if used), must be firmly tightened before fitting.

Note: Two connector tubes (H) replace AR18

Select Installation Position

Mark the bed frame at the 36" dimension

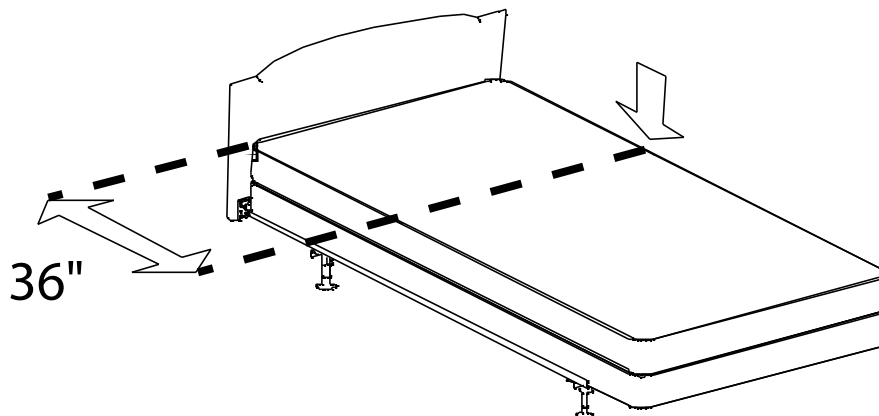


©2005 Hart Mobility, a division of Andrew J. Hart Enterprises Inc.

All rights reserved. Information in this document is subject to change without notice. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Hart Mobility. Other products and company names herein may be the trademarks of their respective owners.


272AR25 Rev. 1 14/01/07

Fitting Arcorail Base to Frame


- 1 Put the ANCHOR ASSEMBLY under the bed frame 36" from the head of the bed.

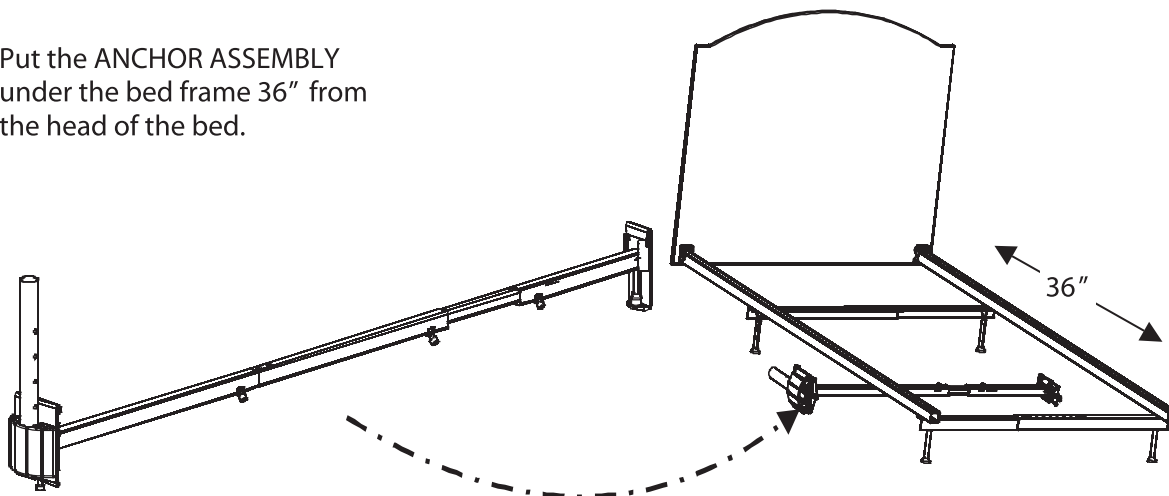

- 2 Hook anchor handle into bed frame rail as shown




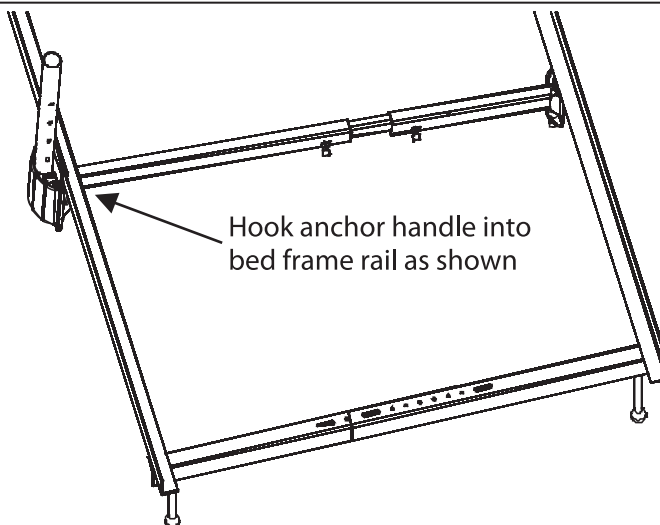


- 3 Take opposite end and hook it onto the bed frame. Proceed to tightening bolts in step 4.

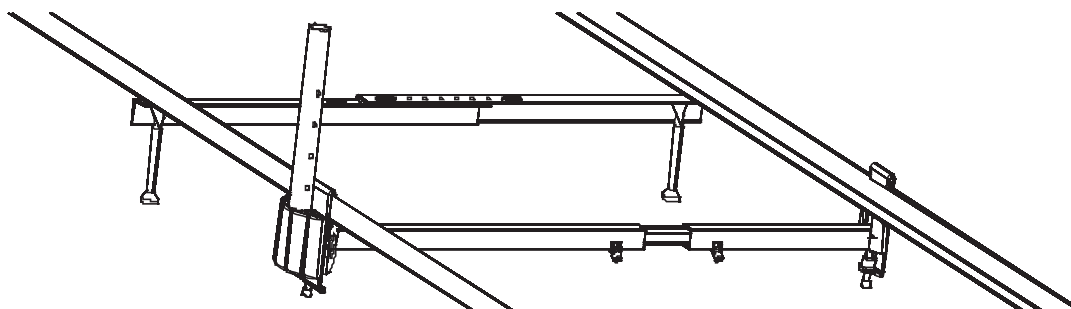


Tightening Arcorail Base to Frame

Ensure all parts are attached firmly (to simulate one solid part) and install in numeric order as shown.

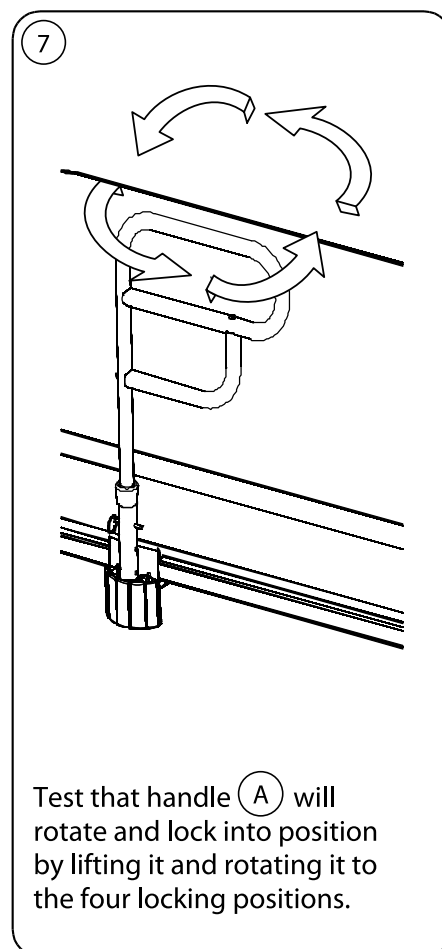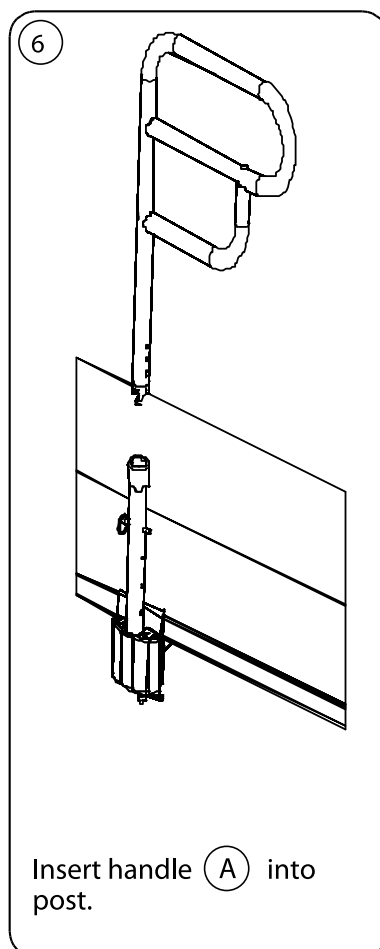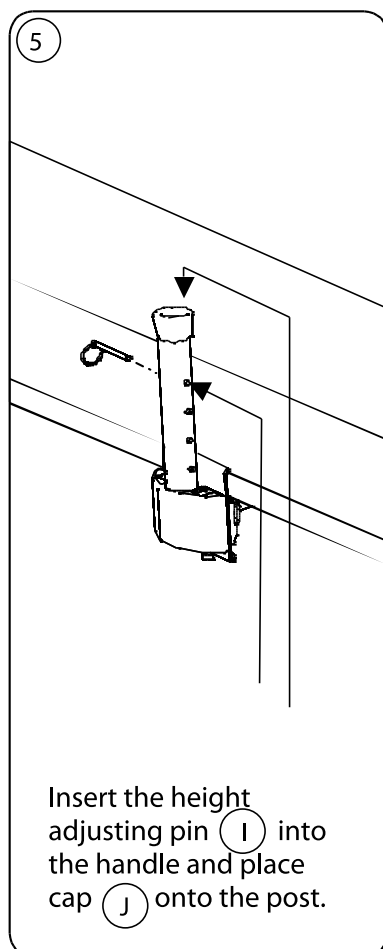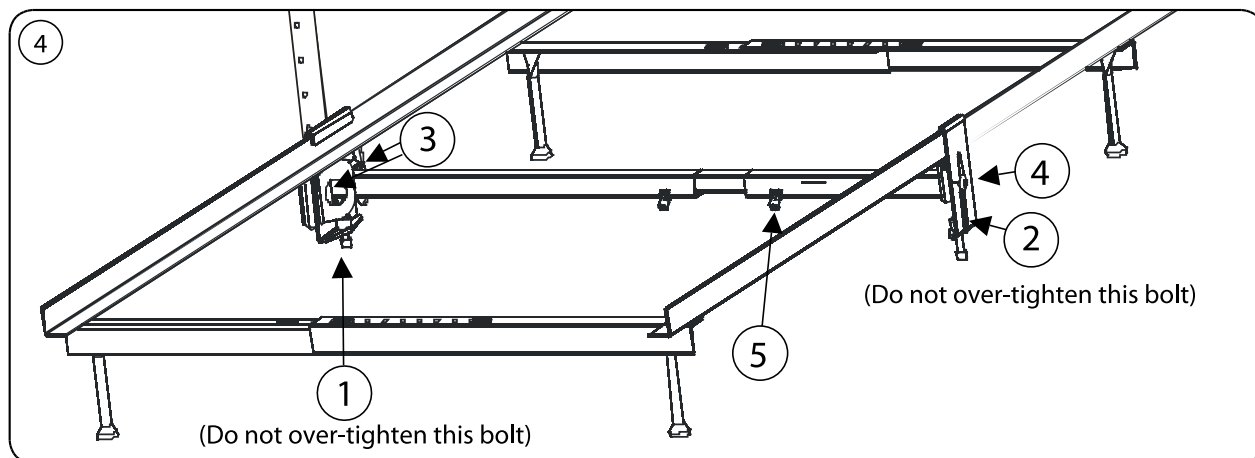

Safety-Check:

Set height of Arcorail for safety (do not allow a gap greater than 3 inches – see below for important details) and check to ensure all bolts and parts are firmly fastened before placing mattresses onto the bed frame and periodically tighten as required.

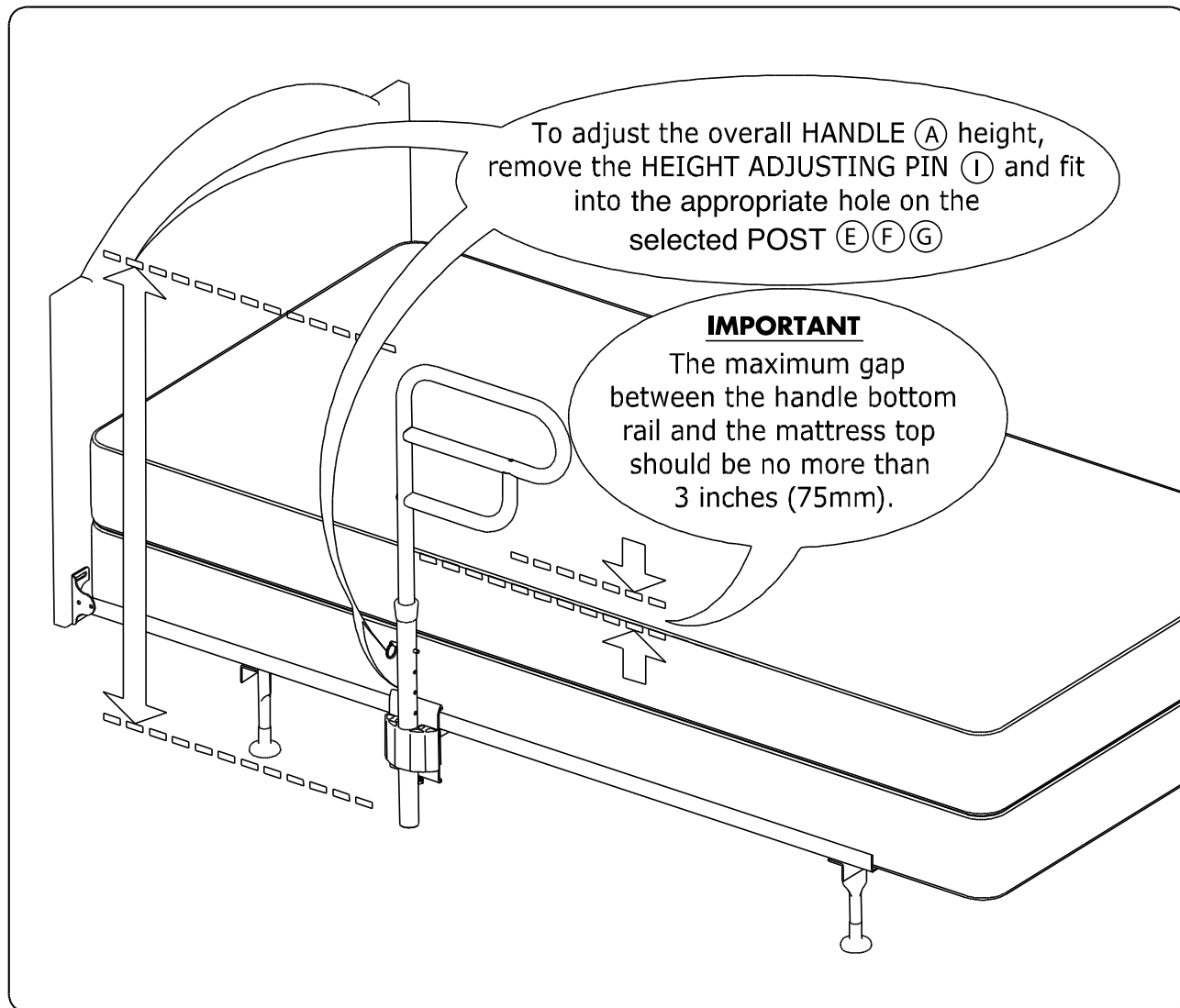


©2005 Hart Mobility, a division of Andrew J. Hart Enterprises Inc.

All rights reserved. Information in this document is subject to change without notice. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Hart Mobility. Other products and company names herein may be the trademarks of their respective owners.

272AR25 Rev. 1 14/01/07


Arcorail Assembly Instructions for HOME-CARE (AR30) and HOSPITAL STYLE (AR40) beds

(AR 30)
FITS HOME-CARE
BED FRAMES
(1" - 3") IN HEIGHT

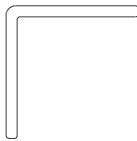

HOME CARE STYLE beds have
INVERTED ANGLE IRON FRAMES that
are horizontal and flat at the top.

(AR 40)
FITS HOSPITAL STYLE
BED FRAMES
(1" - 3") IN HEIGHT

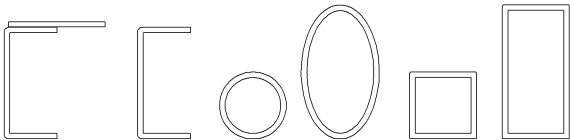


HOSPITAL STYLE beds have a
variety of frame cross-sections.

Please read the following instructions before you start to assemble the Arcorail bedside handrail.
This document contains important safety information.


Bag Contents

Item	Qty.	Description
P	1	LARGE HOME-CARE AND HOSPITAL STYLE CLAMP
Q	1	SMALL HOME-CARE AND HOSPITAL STYLE CLAMP

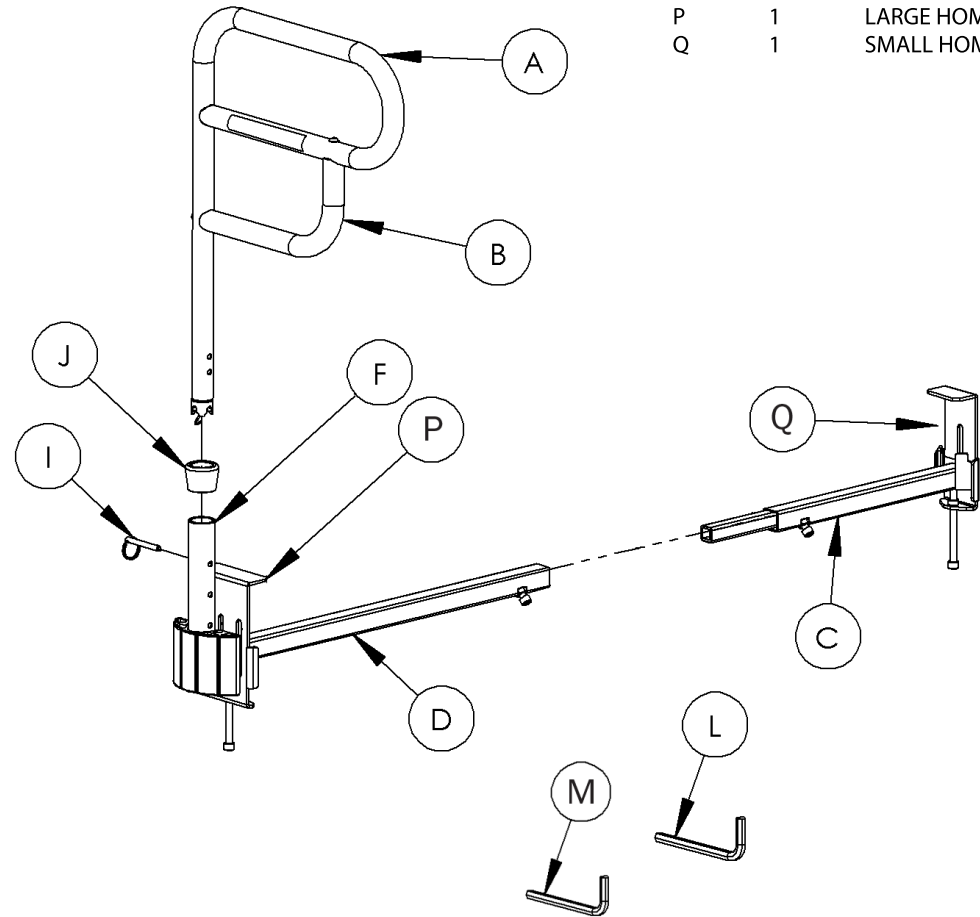

©2005 Hart Mobility, a division of Andrew J. Hart Enterprises Inc.
All rights reserved. Information in this document is subject to change without notice. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Hart Mobility. Other products and company names herein may be the trademarks of their respective owners.
272AR30-40-U Rev.1 14/01/07


1.

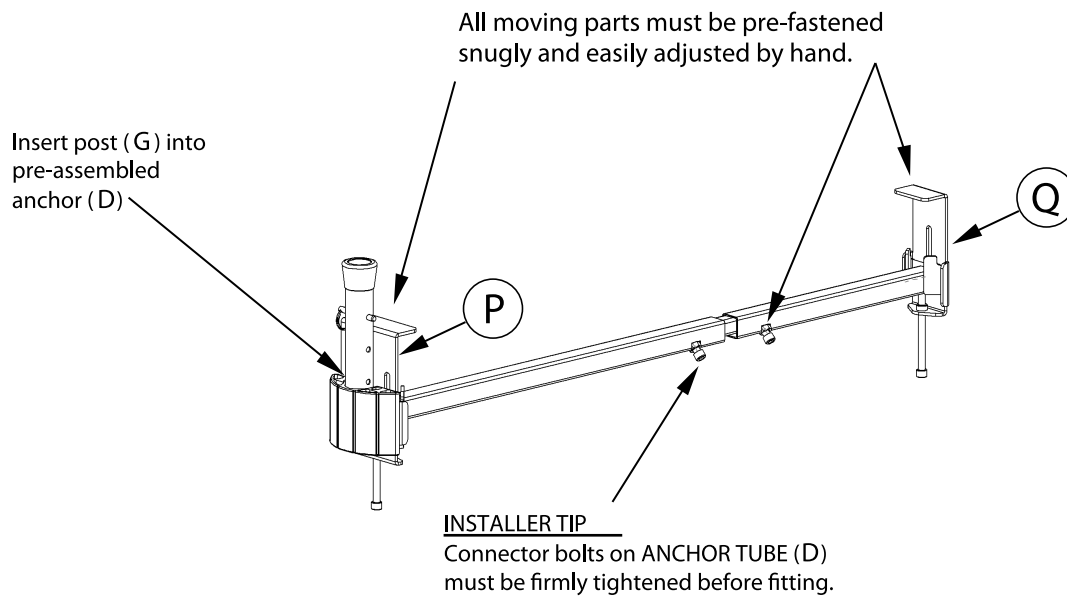

2.

1. Place the FRONT ANCHOR ASSEMBLY approximately 36 " from the head of the bed and onto the bed frame as shown.


2. Place the BACK ANCHOR ASSEMBLY the same distance from the head of the bed and onto the bed frame as shown.

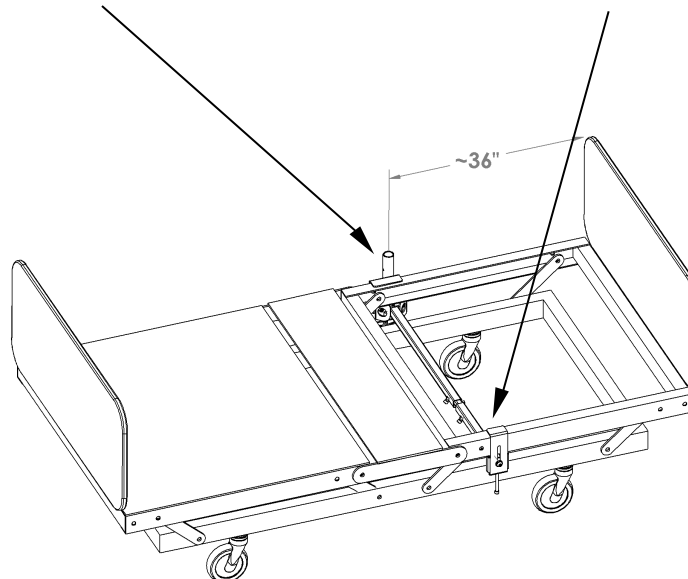


Tightening Arcorail Base to Frame

Ensure all parts are attached firmly (to simulate one solid part) and install in numeric order as shown.

3.

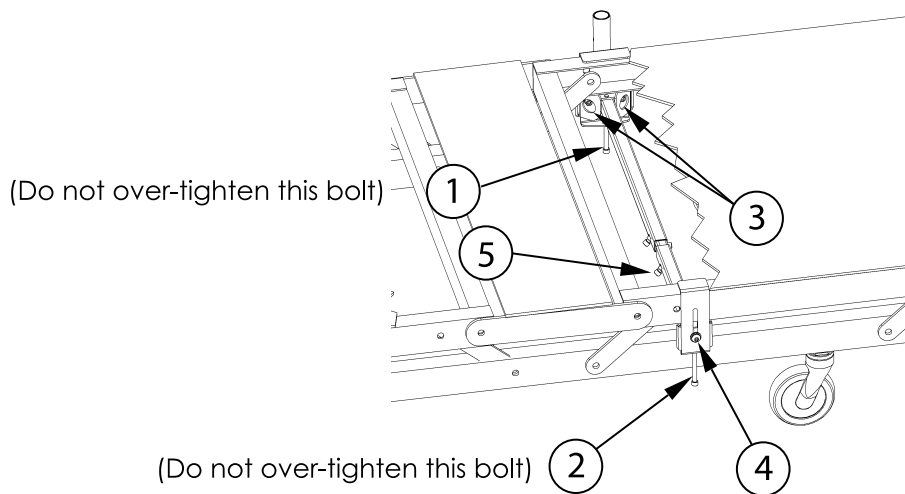


4.

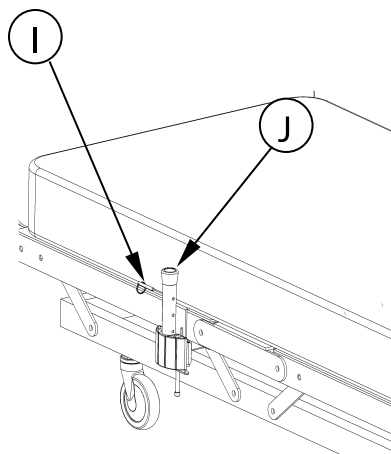

Insert the HEIGHT ADJUSTING PIN (I) into the handle and place CAP (J) onto the top of the post.


5.

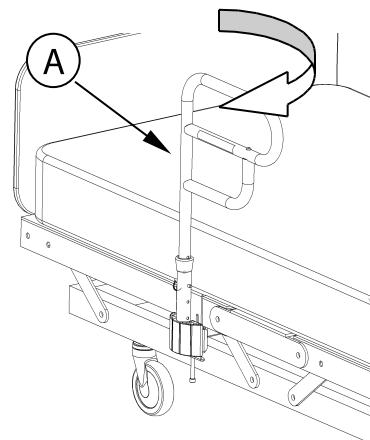

Insert HANDLE (A) into post and test that it will rotate and lock into position by lifting and rotating it to the four locking positions.

Safety-Check:

Set height of Arcorail for safety (do not allow a gap greater than 3 inches – see below for important details) and check to ensure all bolts and parts are firmly fastened before placing mattresses onto the bed frame and periodically tighten as required.

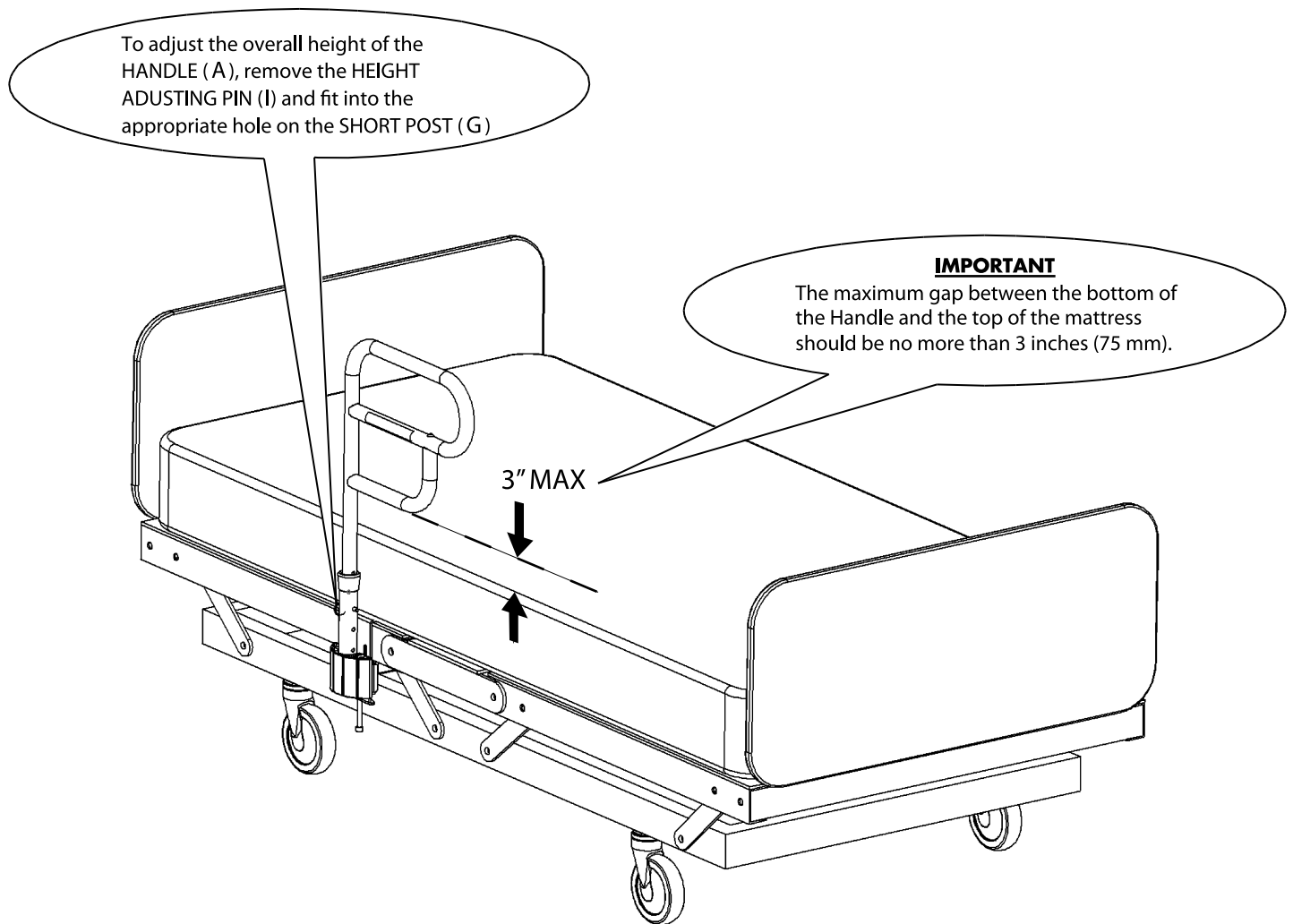

©2005 Hart Mobility, a division of Andrew J. Hart Enterprises Inc.
All rights reserved. Information in this document is subject to change without notice. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Hart Mobility. Other products and company names herein may be the trademarks of their respective owners.
272AR30-40-U Rev.1 14/01/07

Instructions to Convert The Pre-Assembled HOME STYLE (AR25) ARCORAIL to a HOME-CARE or HOSPITAL STYLE (AR30 & AR40) ARCORAIL

Step 1:

Remove the extra parts from the bag marked
"HOME-CARE and HOSPITAL STYLE BEDS (AR30 and AR40) parts"

- Item (P) is the Large HOME-CARE and HOSPITAL STYLE Clamp
- Item (Q) is the Small HOME-CARE and HOSPITAL STYLE Clamp

Step 2:

Disassemble the Clamps: Item (N) on Assembly (D) and Item (O) on Assembly (C).
Replace these two Clamps with Items (P) and (Q) from the Bag.

Step 3:

HOME-CARE and HOSPITAL STYLE beds only use the Short Post Item (G).
Discard the Long Post – Item (E) and the Medium Post – Item (F).

Step 4:

The two pieces of Item (H) are not required – discard them also.

Step 5:

You have converted the ArcoRail and now follow the Installation Instructions
for the HOME-CARE and HOSPITAL STYLE (AR30 and AR40) beds.

