

# Care Bag Travel kit



Reorder number: **9538557**

**Presentation** This travel kit is designed for kids and adults who suffer from transportation sickness. It features a hygienic bag for urines and vomit and a refreshing, cleaning wipe. Being pocket-sized, it is a must-have for handbags and glove compartments.

**User's Manual**

1. Open the pack and take a bag
2. Unfold the bag
3. Open the bag and check the presence of the absorbent pad
4. Hold the bag with your hands
5. Vomit into the bag
6. Wait for the solidification of the liquids (+/- 500ml max in a few seconds)
7. Close the bag, seal it, and dispose of in the appropriate place
8. Use the freshen-up cleansing wipe

**Precaution of use**  Single-use product. Do not re-use

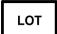

**Product and material** Polyethylene bag equipped with threads. Super-absorbent pad (SAP) made of a non woven fluff coupled with copolymers made of sodium acrylate.

Wipe: aqua, trimethylpopane, tricaprylate/tricapate, ceteth 24, diazolidinyl urea, dimethicone ; PEG 100 Stereate, choleth 24, glycerin, cetareth-20, ctearyl alcohol, glyceril stearate, perfume, chlorhexidine dicgluconate 20%

**Efficiency** Absorbs 500 ml of organic liquids or several litres of demineralised water


**Benefits**

- No more vehicle or bed cleaning
- Suppresses smells
- More hygienic, single-use product, time-saving, blocks smells, easy to dispose of without any risks

**Identification** Each product is identified by a reference number  and an expiry date  (24 months).

**Conformity** Compliant with the 93/42/CEE regulation on Class I medical device  
This product fits in anti-pollution legislation in terms of incineration risks.  
The incineration does not release any chloro element or any by-product that might create dioxine



**Dangers/safety** This product is not considered dangerous according to the European Union regulation  
 Very light risks of ocular or respiratory tracts irritation in case of prolonged contact with the absorbent pad.

To be kept in its closed pack away from any source of humidity  
Safety datasheet available on request

**Customs code** 5603149090

<b>Packing</b>	Unit	Pack of 3 travel bags	57/8x 13/16x315/16 / 0.22 lbs
	Display	8 packs	203/16x1013/16x43/4 / 660.00 lbs