DOLOMITE

DOLOMITE *HP*

SV BRUKSANVISNING

NO BRUKSANVISNING

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GB OPERATING INSTRUCTIONS

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FR MANUEL D' UTILISATION

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DOLOMITE HP

SV BRUKSANVISNING

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GB OPERATING INSTRUCTIONS

Study the operating instructions carefully **before** use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

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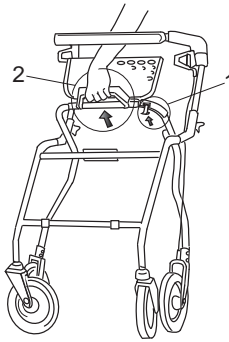
1. Lyft upp lyftspärren.
2. Fäll ihop.

GB FOLDING

1. Lift up the safety latch.
2. Start the handle to fold the walker.
Folded position.

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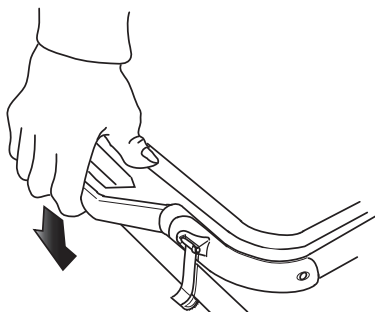
GB OPERATING POSITION

For walking position, lift up handle press down to lock with thumb. A clear sound indicates correct position.

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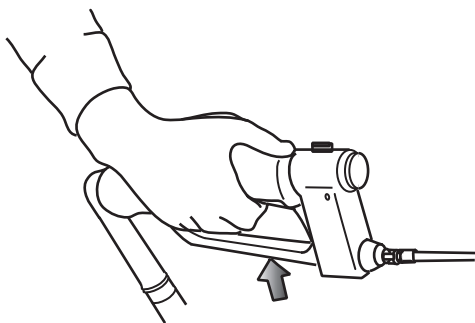
GB PARKING BRAKE

Press down carefully for parking brake. A clear sound is heard when the brake is placed in correct position. N.b. Always place both brakes in parking position. Pull upwards to loosen the brake.

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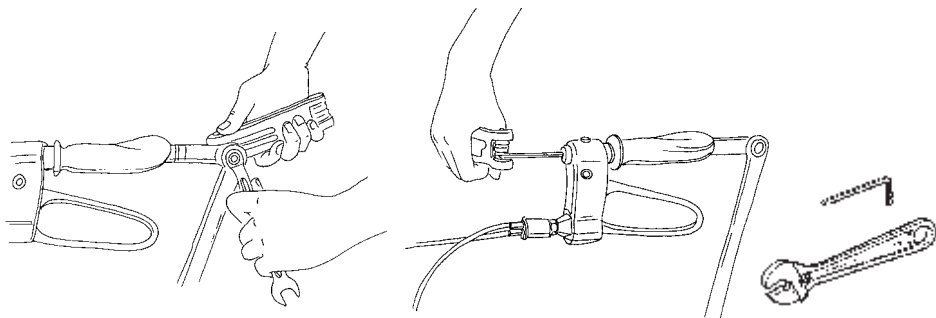
GB BRAKE

Place the handles in desired angle and tighten properly with an allen key and a wrench. Grip and brake handle can be rotated to suit the hands position. Adjust to required position and lock with allen key. **Suitable tools (not included).**

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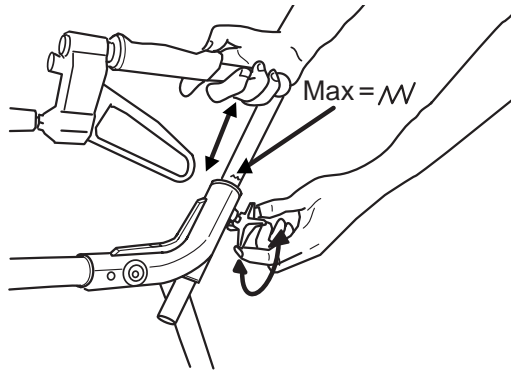
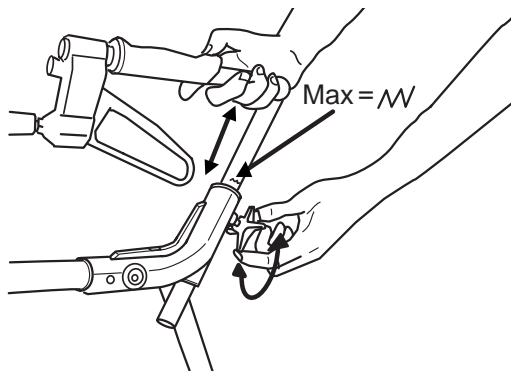
DOLOMITE HP

GB HANDLEBARS

For adjustment; Loosen the wing, adjust the handle to suitable height and tighten the wing properly. Therapist may suggest different heights depending on specific need of the client.

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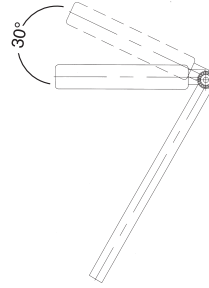
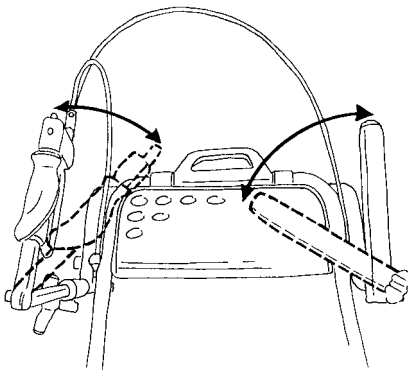
GB HANDLEBARS

The handles can be angled maximum 45° towards the center of the walker. Arrow pointing on the thin inner vertical line indicates maximum allowed outer handle position. Handles should be placed horizontally maximum 30° upwards.

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Håndtak kan vinkles maksimum 45 grader innover. Håndtaket er innstilt i maksimum. Når pilen peker på den indre tynne vertikale linje, indikeres maks tillat ytre håndtaksstilling. Håndtakene kan vinkles maksimum 30° oppover fra horisontal stilling.

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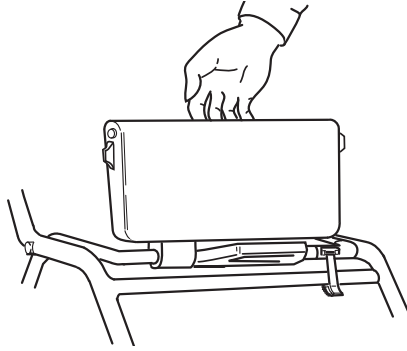
GB FOLDABLE SEAT AND FOLDING

The seat is foldable for increased space between the handles.

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All technical data available on the DOLOMITE HP plus label:

- | | | |
|-----------|--------------------|---------------------------------------|
| GB | 1: Article number | 4: Date of manufacture |
| | 2: Name of product | 5: Maximum user's weight |
| | 3: Serial number | 6: Attention: see instruction for use |

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GB**INSTRUCTIONS FOR DESTRUCTION OF PRODUCT AND PACKING:**

When the product can no longer be used, it and its accessories as well as the packaging material should be assorted in the correct way. The individual parts can be separated and handled according to the list specification.

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GB MAINTENANCE – SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments** are properly tightened.
- Lubricate the wheels regularly with grease.
- **Immediately contact the nearest dealer if a fault should appear. Never use a defective walker.** Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid! User's max. weight: 275 lbs. (125 kg, 20 stones.) DOLOMITES will not accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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DOLOMITE AB
AQUATEC GmbH & Co.KG
MOBITEC Mobilitätshilfen GmbH