

Dolomite® Maxi+ ®

en	Rollator User Manual	1
fr	Déambulateur Manuel d'utilisation.....	5

General

About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.



WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.



CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.



IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.



Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.



This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Intended use

Indications

The rollator is intended for support and improvement of a restricted walking ability. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others.

The rollator is intended to support a person during walking and may also serve as a resting seat. Furthermore the rollator is intended to carry a load either in a specially designed basket or on a tray.

The max. user weight is 205 kg / 452 lbs. This weight limitation includes the weight of the user whether standing or sitting and the content of the basket.

Contraindications

The Rollator is not intended for use by persons with severe equilibrium disorders or functional perceptual disorders.

Permissible operating conditions and locations

The rollator is only intended for use indoor in living or public areas or outdoor. The rollator is not to be used under water in e.g. pools or open-air bath, neither in hot areas as e.g. saunas.

Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance




This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.



Yes, you can.®

intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

Scope of delivery

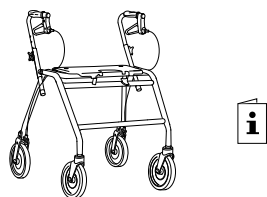

- Rollator, fully assembled.
- User manual

Accessories, actually assembled to your rollator, are depending on the configuration and might differ per region.

Limited Warranty — North America

Limited Lifetime Warranty. Product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic and routine replacement. Maintenance and operating instructions must be followed for the warranty to be valid.

Safety

General Warnings



WARNING!

Risk of falling

- Exercise caution when walking on sloping / slippery ground and on uneven surfaces.
- Do not use the rollator as a wheelchair.
- Be careful when walking rearwards over obstacles.
- Never sit and push with your feet.
- Do not go up or down stairs with the rollator.
- Do not use the rollator on escalators.
- Do not under any circumstances stand on the rollator.
- Be careful when using long skirts, the clothes might get stuck in the wheels.
- Loads must only be transported in the basket or on the tray. Be careful when there is a heavy load in the basket (for the max. load in the basket refer to chapter “Technical data”).



WARNING!

Risk of injury

- Do not let children play with the rollator.
- If the push handles or the seat are exposed to extreme temperature (above 41°C / 106° F) there might be risk for first degree burns.




IMPORTANT!

Risk of corrosion

- The rollator must not be stored outside in wet climate or in humid environments for a long period of time.

Labels and symbols on the product

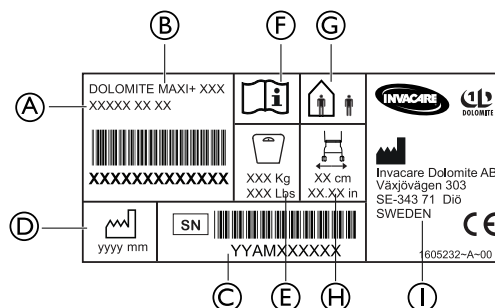

The label contains all important technical information regarding your walking aid.

- | | |
|-----------------------|-------------------------------|
| Ⓐ Article number | Ⓕ Refer to user manual |
| Ⓑ Product name | Ⓖ In- and outdoor walking aid |
| Ⓒ Serial number | Ⓗ Max. width of walking aid |
| Ⓓ Date of manufacture | Ⓘ Manufacturer's name. |
| Ⓔ Max. user weight | |

Usage

Before use

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.


Folding and unfolding the rollator

Unfolding to operating position




WARNING!

- Always make sure that the rollator is fully unfolded and the lock is engaged before using the rollator.

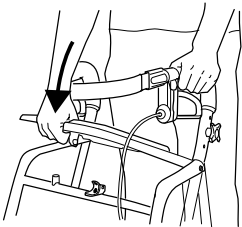

1. Press down the seating tube.
2. Check that the safety catch is in the locked position.


Folding

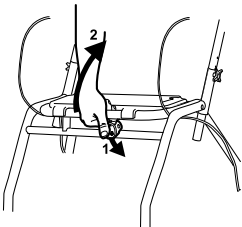

1. Press down the safety catch.
2. Pull the seating tube upwards.



Flip up the seat plate

You can flip up the seat to provide a larger walking space and to increase accessibility.

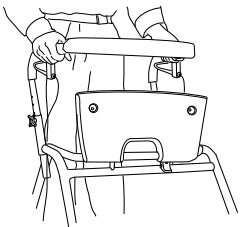

Using the hand brakes

- 1.
- 2.

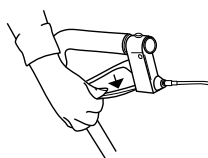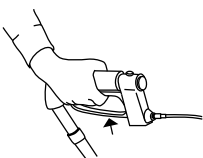

1. Apply the brakes by pulling both brake handles upwards.
2. To apply the parking brakes, press the brake handles carefully downwards.


An audible sound indicates that the brake is in the correct position.



It is important that both brakes are put into parking position.

3. To release the brake, pull the brake handle upwards.


Adjusting the brake effect

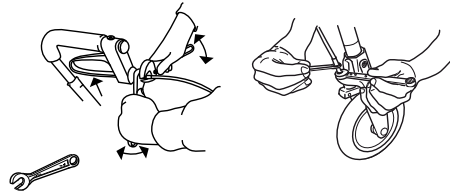

For desirable braking strength, adjust the brake wire.

Suitable tools: 8 mm / 0,3 inches 10 mm / 0,4 inches spanner (not included).


Adjusting the handle height

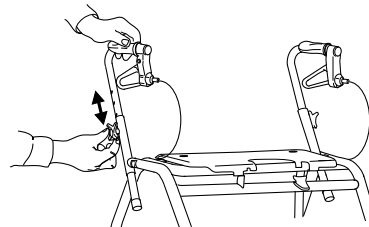

1. Stand behind the rollator with flat shoulders.
2. Loosen the knob completely, remove the screw.
3. Adjust the handle to suitable height.



The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.

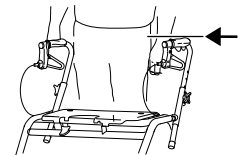

4. Fix the screw into the hexagonal hole.
5. Fasten the knob properly.
6. Repeat the procedure on the other side.


Walking



WARNING!


Risk of falling

- Be careful if the wheels stop against an obstacle.

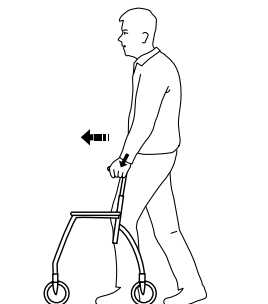

1. Place the walking aid slightly in front of you and take a step towards it while using the hand grips for support.

Sitting down on a chair

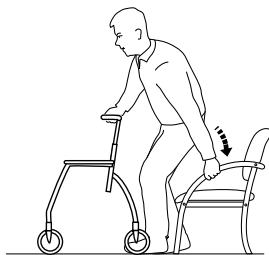

1. Stand in front of the chair with the walking aid, the back of your legs should touch the chair.
2. Place your hands on the arms of the chair, lean forward slightly and sit down carefully.


Rising to a standing position



WARNING!

Risk of tipping

- Never use the walking aid to pull yourself up from a sitting position.
- The parking brakes must be in braked position when the walking aid is used as a support when sitting down or getting up, from a chair for example.

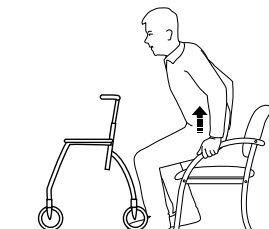

1. Place the walking aid in front of you, use the armrests of the chair as support and rise carefully to a standing position.
2. Once stable in a standing position, transfer hands to the hand grips of the walking aid.

Sitting on the walking aid



WARNING!

Risk of falling

- Use the walking aid for stationary seating only. Both brakes must be in parking position when the walking aid is used as a seat.
- Never sit and push with your feet.
 - Never sit on the walking aid on sloping ground.

Accessories

Accessories are not standard on all models. For a list of available accessories, contact your authorized dealer or the Invacare representative (see rear page of this manual).

Basket

Mount the basket by hanging it with the hooks on the front frame.

Tray

Secure the tray by using the plastic plugs.

Maintenance and care

Maintenance



Maintenance should be done at least once a year.

To ensure good performance and safety you should regularly check that the following is carried out:

- Ensure that screws and adjustments are properly tightened.
- Never use a defective walking aid. Do not attempt repairs yourself!
- If hand grips are loose do not use the walking aid. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Verify brake operation. If brake adjustment is required, see chapter Usage, page 3 in this user manual.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

Cleaning

The product can be cleaned and disinfected using commercially available agents.



IMPORTANT

- Cleaning must be done at least once a year.
- Do not use an abrasive detergent cleaner.
- Do not use a steam cleaner.
- Never use acids, alkalies or solvents.

- The product can be washed in a washing machine or by hand.
- The washing and drying temperature may not be higher than 60° C / 140° F.
- High-pressure cleaner or ordinary jet of water may be used. However, do not aim the jet towards bearings, draining holes or labels.
- Clean with damp cloth and mild detergent.
- Clean the wheels with a brush (no wire brush!).
- After washing, wipe the product dry.

Disinfection

Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

After Use

Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way.

The individual parts can be separated and handled according to their material.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Re-use

This product is suitable for re-use after the following has been performed:

- Inspection
- Cleaning


For details, see chapter Maintenance and care, page 4 .



WARNING!


Risk of injury

- Always inspect the product according to the service instructions before transferring the product to a different user.
- If any damage is found do not use the product.

 The user manual belongs to the product. Make sure that the user manual will be handed over with the product.

Technical data

Dimensions and weights

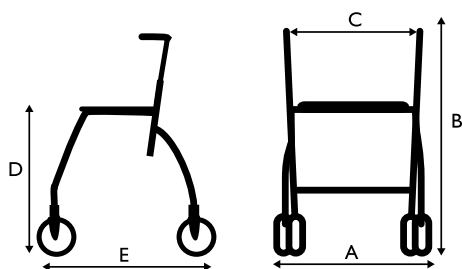

	Maxi+ 550	Maxi+ 650
Total width (A)	700 mm / 27,56"	
Total height (B)	725 mm / 29" – 895 mm / 35"	815 mm / 32" – 985 mm / 39"
Seat width (C)	520 mm / 20"	
Seat height (D)	555 mm / 22"	650 mm / 26"
Total length (E)	640 mm / 25"	740 mm / 29"
Front/rear wheel ¹ diameter	200 mm / 8"	
Front/rear wheel ¹ width	40 mm / 2"	

¹ Front wheels are steering wheels. Rear wheels are provided with a brake system for dosed breaking and for locking the wheel.

	Maxi+ 550	Maxi+ 650
Weight ²	9,8 kg / 21,6 lbs	10,3 kg / 22,7 lbs
Max. user weight ³	205 kg / 452 lbs	
Max. load basket	9 kg / 20 lbs	
Max. load tray	5 kg / 11 lbs	

² Total weight excluding equipment.

³ This weight limitation includes the weight of the user whether standing or sitting and the content of the basket.

Material

Frame	Polyester powder coated steel
Handle bars	Chrome plated steel
Handgrips	TPE
Wheels	Polypropylene with treads of TPE
Plastic parts	Material according to marking (PA, PP, PE)
Screws	Steel, rust protected

Symboles présents dans ce manuel d'utilisateur.

Dans ce manuel, les avertissements sont indiqués par des symboles. Les symboles d'avertissements sont accompagnés d'une mention sur l'importance du danger.



AVERTISSEMENT!

Indique une situation dangereuse qui pourrait causer des blessures graves, voire mortelles, si elle n'est pas évitée.



MISE EN GARDE!

Indique une situation dangereuse qui pourrait causer des blessures mineures ou légères, si elle n'est pas évitée.



IMPORTANT

Indique une situation dangereuse qui pourrait causer des dommages matériels si elle n'est pas évitée.



Conseils et recommandations

Indique des conseils, des recommandations et des renseignements utiles pour une utilisation efficace et sans problème.



Ce produit est conforme à la directive 93/42/CEE relative aux dispositifs médicaux. La date de lancement de ce produit est indiquée dans la déclaration de conformité de la CE.

Généralités

À propos de ce manuel

Ce manuel d'utilisateur comprend des renseignements importants au sujet de ce produit. Afin d'assurer une utilisation sécuritaire, lisez le manuel d'utilisateur attentivement et suivez les consignes de sécurité.

La personne qui prescrit le produit ou qui est responsable de l'utilisateur, doit s'assurer qu'il est adapté à ce dernier. Toutes les instructions doivent être données à l'utilisateur du produit. Cela concerne aussi bien l'utilisation des fonctionnalités que les restrictions d'utilisation. Les mises en garde du manuel d'utilisateur doivent également être mentionnées.

USA:

Clarke Health Care Products
7830 Steubenville Pike
Oakdale PA 15071
888-347-4537
www.clarkehealthcare.com

Canada:

Invacare Canada LP
570 Matheson Blvd E. Unit 8
Mississauga Ontario
L4Z 4G4, Canada
Phone: (905) 890 8300
Fax: (905) 501 4336

Manufacturer:

Invacare Rea AB
Växjövägen 303
SE-343 71 Diö
Sweden

1587618-D 2022-09-30



Making Life's Experiences Possible®



Yes, you can.®