SAVE THESE INSTRUCTIONS

Yes, you can.[®]

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

⚠ WARNING

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- You should **ALWAYS** consult with your physician or therapist to determine proper adjustment and usage.
- Items should not protrude from the walker tray - otherwise, injury or damage may occur.
- After installation, ensure that the walker and accessories are properly and securely locked in place and hardware tightened securely before using.
- **DO NOT** carry or place weight exceeding five pounds in or on the walker tray when attached to walker. This may cause the walker to tip forward.
- **NEVER** use the walker tray, walker basket and the walker pouch at the same time.
- **NEVER** use the walker tray as a means of support.
- **DO NOT** sit or push down on the walker tray when in use. This may cause the tray to collapse.
- **DO NOT** use with heavy duty walkers.

⚠ ACCESSORIES WARNING

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Installing the Tray Side Braces

NOTE: Before installing tray side braces onto the walker for the first time, soak them in hot water for five minutes.

1. Make sure the openings in the tray side braces are pointing in towards the center of the walker.



NOTE: The tray side braces attach differently on dual release walkers than single release walkers.

2. Perform one of the following:
 - Dual Release Walkers - Position the openings in the tray side braces above and below the pivot brace (Detail A).
 - Single Release Walkers- Position both openings in the tray side braces above the pivot brace (Detail A).

3. Snap the tray side braces onto the walker at the points determined in Detail B.


NOTE: If necessary, use a rubber mallet to snap the tray side braces onto the walker.

4. Perform one of the following:
 - Dual Release Walkers - Proceed to Installing the O-rings onto the Tray Links.
 - Single Release Walkers - Proceed to Installing Snap Supports - Single Release Walkers Only.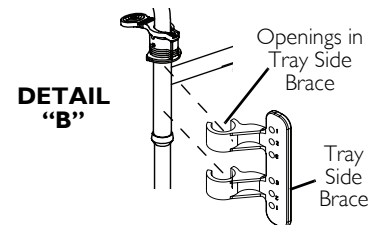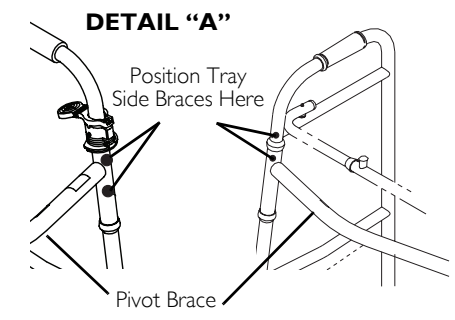


Installing Snap Supports - Single Release Walkers Only

1. Locate the upper hole in the tray side brace that is marked "2".
2. On the inside of the tray side brace, attach the snap support into the location determined in STEP 1.
3. Repeat for opposite tray side brace.
4. Proceed to Installing the O-rings onto the Tray Links in this instruction sheet.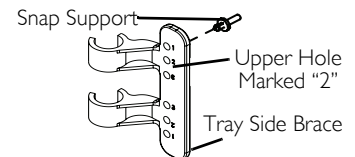

Installing the O-rings onto the Tray Links

1. Install the four O-rings over the ends of the walker tray links.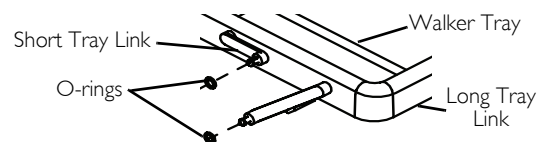

Attaching the Walker Tray to the Tray Side Braces

1. Place the walker tray onto the walker with the walker tray links positioned towards the front of the walker.
2. Insert and lock the tray links into the following holes of the tray side brace.


NOTE: There are two sizes of tray links. Refer to the chart on page 2 to determine tray link size.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com


SHORT TRAY LINK POSITIONING		LONG TRAY LINK POSITIONING	
FOR DUAL RELEASE WALKERS	FOR SINGLE RELEASE WALKERS	FOR DUAL RELEASE WALKERS	FOR SINGLE RELEASE WALKERS
SECOND HOLE FROM TOP (MARKED "2")	TOP HOLE (MARKED "1")	FIFTH HOLE FROM TOP (MARKED "2")	FOURTH HOLE FROM TOP (MARKED "3")

DETAIL "A"


NOTE: Single release walker shown for clarity.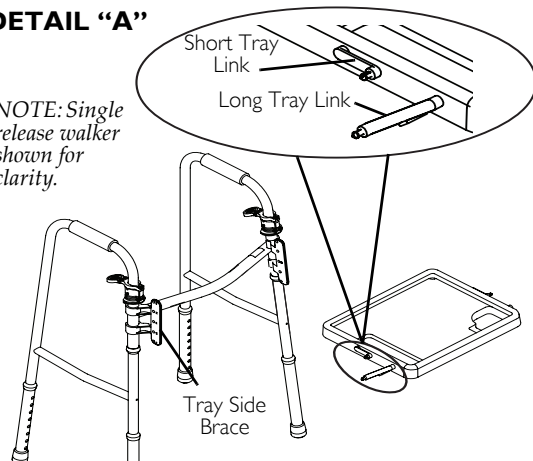

DETAIL "B" DUAL RELEASE WALKERS

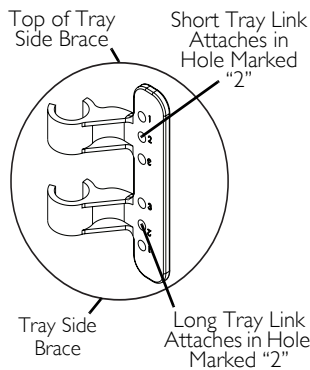DETAIL "C" SINGLE RELEASE WALKERS

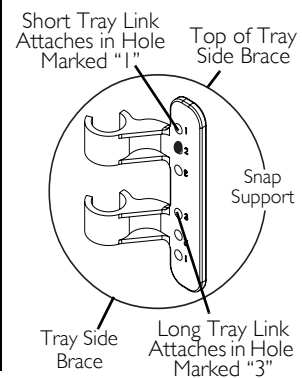Adjusting the Walker Tray


Storing the Walker Tray

1. Pull the back of the walker tray up and then push forward and down on the back of the walker tray to store.

NOTE: The walker tray will lie flat against the walker pivot brace when storing.

Preparing to use the Walker Tray

1. Pull the back of the tray up and pull in on the back of the tray until the tray is level.

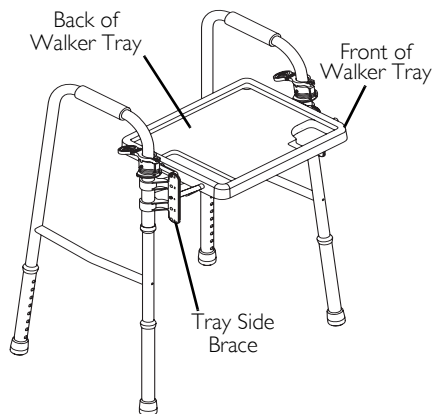Removing the Walker Tray

1. Grasp the bottom of the tray side braces and pull up and out.

Cleaning

NOTE: DO NOT use an abrasive detergent/cleaner or cloth to clean the walker tray.

1. Use only non-abrasive detergent/cleaner with warm water.
2. Rinse with warm water and wipe dry.

Limited Warranty

Invacare warrants this product to be free from defects in materials and workmanship.

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants all walker models to be free from defects in materials and workmanship for one year for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will use a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

CONSERVER CES INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

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All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
© 2008 Invacare Corporation