AMIGO

SV BRUKSANVISNING

NO BRUKSANVISNING

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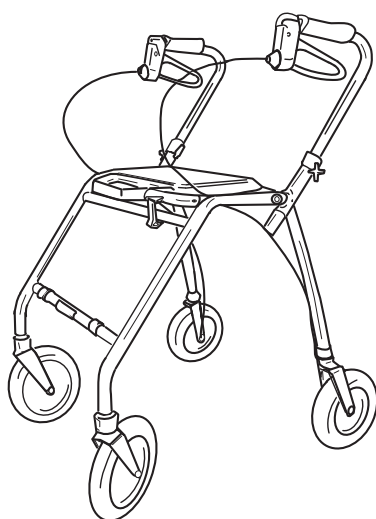
GB OPERATING INSTRUCTIONS

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AMIGO

SV BRUKSANVISNING

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GB OPERATING INSTRUCTIONS

Study the operating instructions carefully **before** use. The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user.

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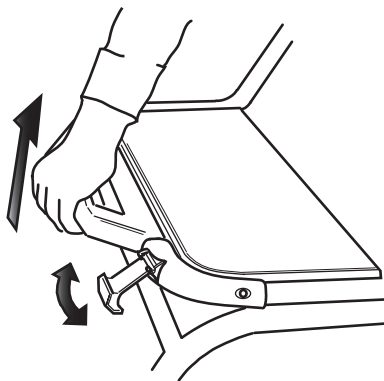
GB FOLDING

Lift up handle to fold. If the walker is equipped with a folding blockage this should be lifted first then the folding handle.

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GB FOLDING

Folded position. Space saving. Suitable for carrying in stairs. Using the parking brake. Make sure that the liftcatch is placed in lowered position. Does not apply to North America

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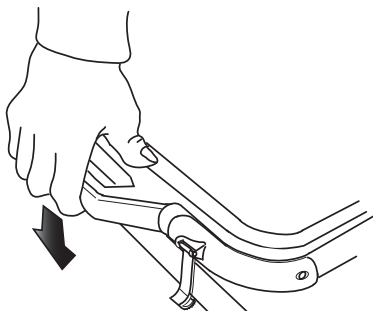
GB OPERATING POSITION

For walking position, lift up handle press down to lock with thumb. A clear sound indicates correct position.

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GB BRAKE

Press down carefully for parking brake. A clear sound is heard when the brake is placed in correct position. N.b. Always place both brakes in parking position. Pull upwards to loosen the brake. Do not use too much power to handbrake the walker. The brakehandle might click twice. This act does not influence the walkers' function.

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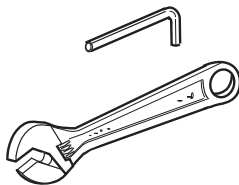
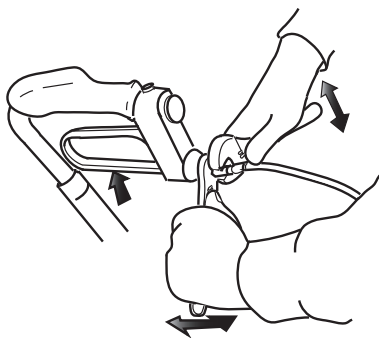
GB BRAKE

For desirable braking strength, adjust the brake wire. Suitable tools (not included).

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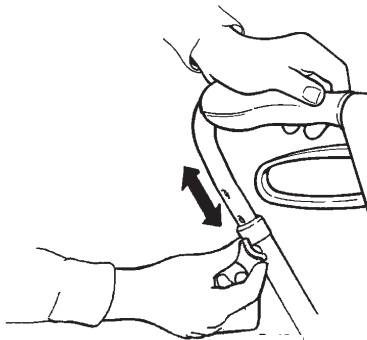
GB HANDLEBARS

Adjust the handle to suitable height. When standing behind the walker with flat shoulders adjust handles to be at the same height as the user's wrist. Therapist may suggest different heights depending on specific need of the client.

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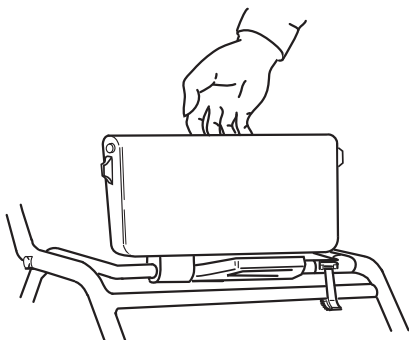
GB FOLDABLE SEAT AND FOLDING

The seat is foldable for increased space between the handles. When the seat is folded up the blockage will drop out of position, this does not effect the function. Does not apply to North America

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GB MAINTENANCE – SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments** are properly tightened.
- Lubricate the wheels regularly with grease.
- **Immediately contact the nearest dealer if a fault should appear. Never use a defective walker.** Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid! User's max. weight: 275 lbs. (125 kg, 20 stones.) DOLOMITES will not accept responsibility for adaptations, faulty repairs, etc, that are outside their control.

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DOLOMITE AB
AQUATEC GmbH & Co.KG
MOBITEC Mobilitätshilfen GmbH