DE

Badewannenlifter

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EN

Bathlift

Operating Instructions

FR

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Mode d'emploi

IT

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ES

Aparato elevador de bañera

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NL

Badlift

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SV

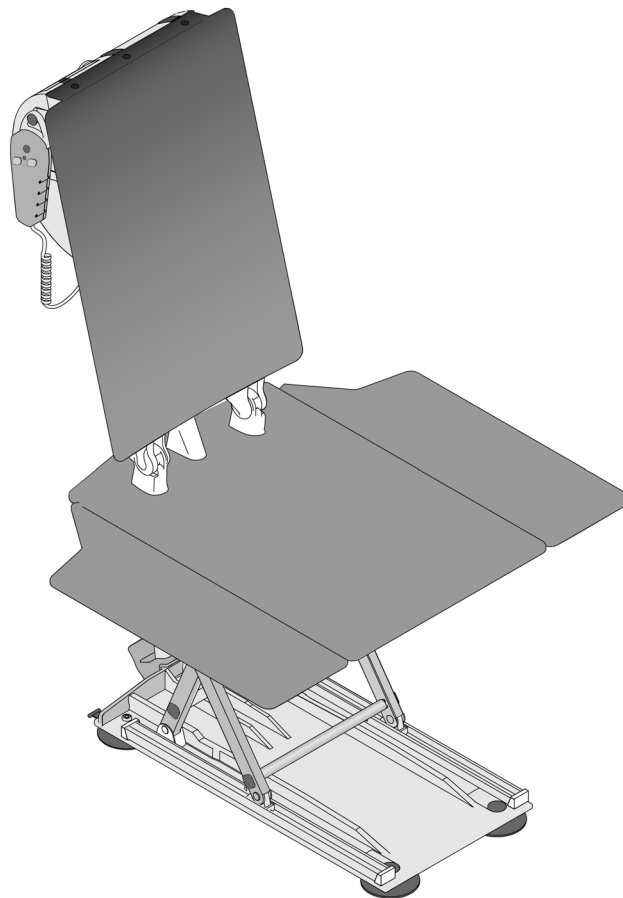
Badkarslyft

Bruksanvisning

NO

Badekarløfter

Bruksanvisning

DE

WICHTIG!

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EN

IMPORTANT!

Before you operate the bathlift **AQUATEC FORTUNA** for the first time, please read the instructions in this manual carefully. In particular, pay attention to the safety instructions and comply with all instructions.

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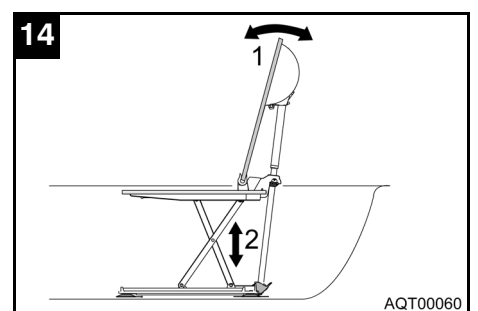
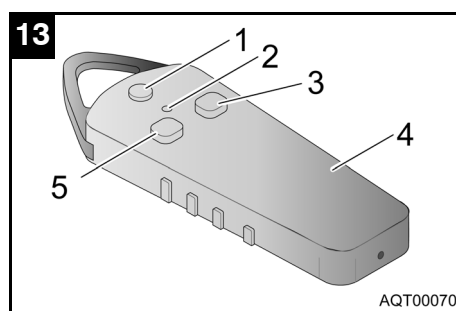
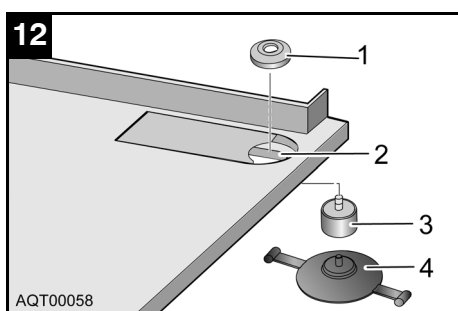
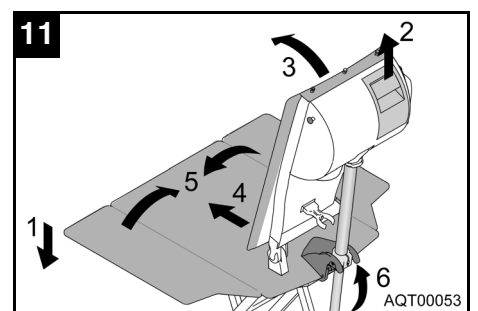
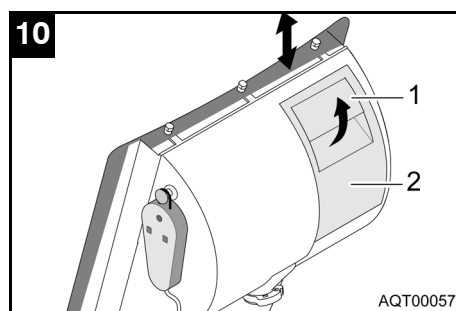
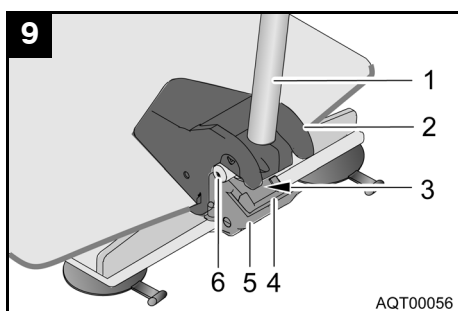
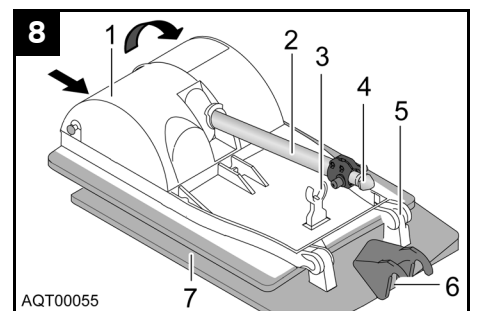
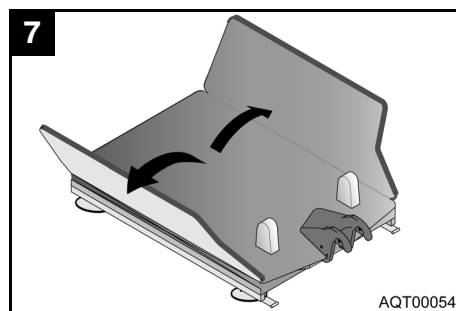
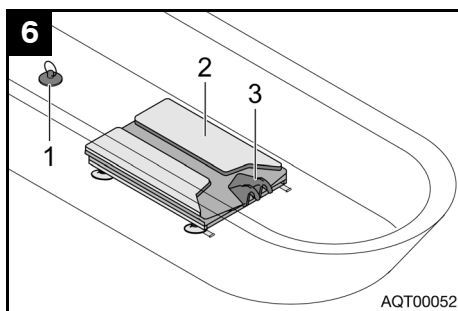
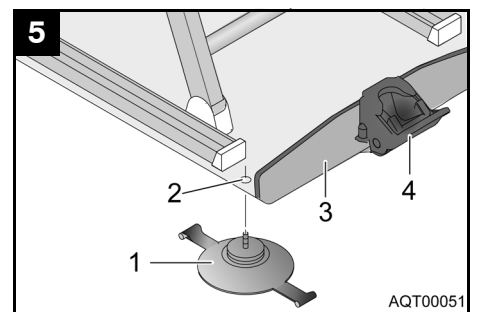
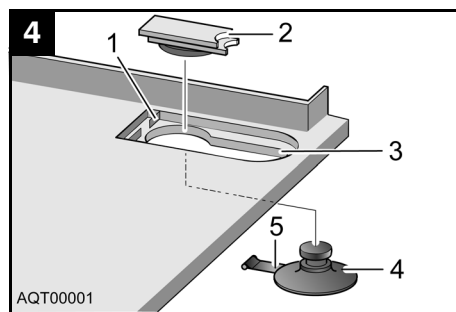
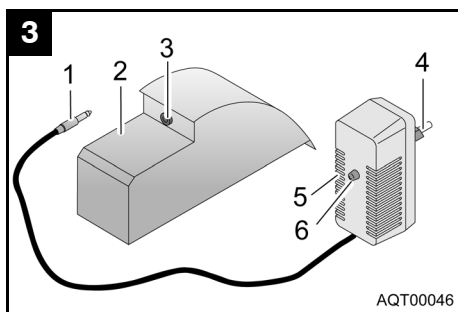
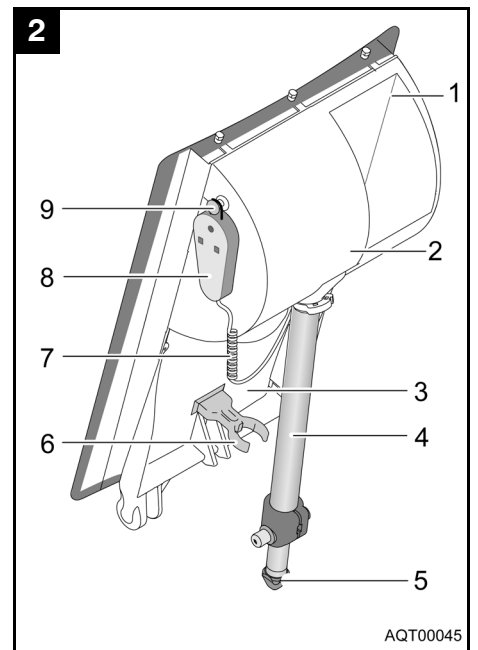
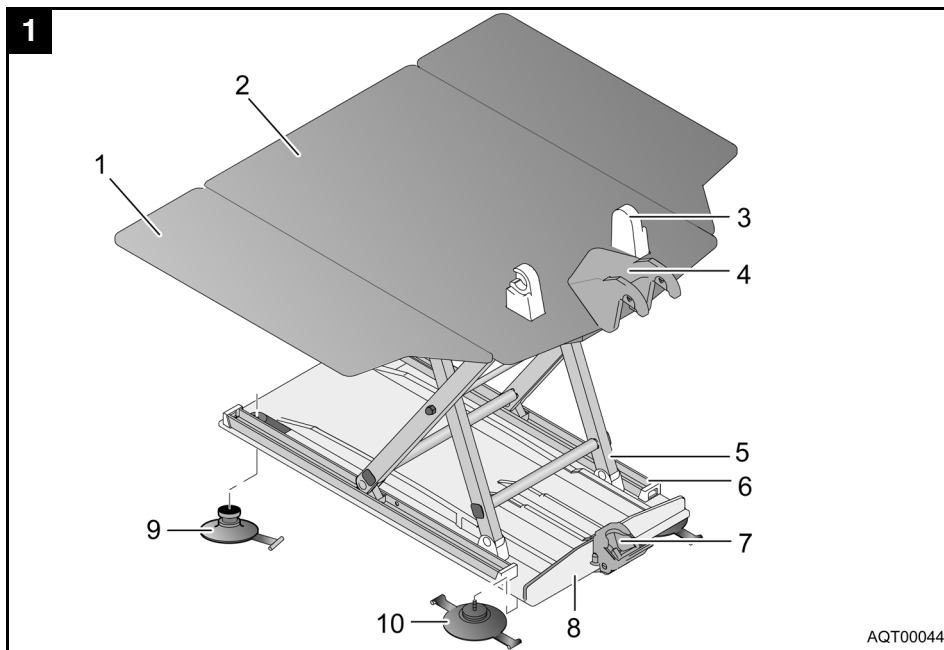
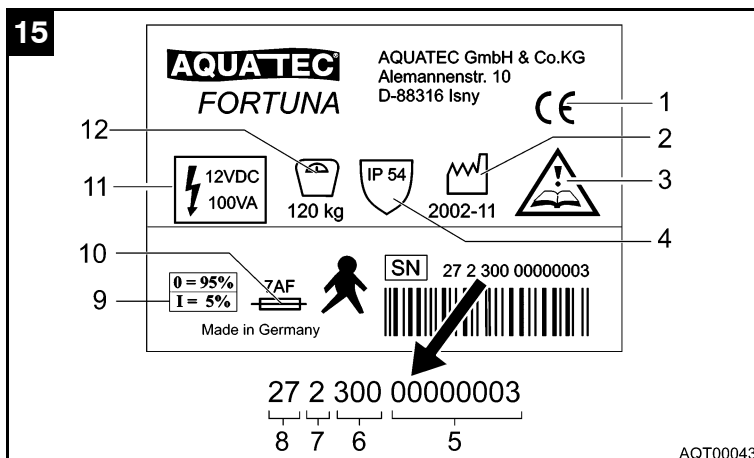
EN

- 1 Declaration of Conformity
- 2 Period of production
- 3 Instruction (read operating manual)
- 4 Protection class
- 5 Serial number
- 6 Day of production (1 to 365)
- 7 Last digit of year of production
- 8 Device type
- 9 Operating factor
- 10 Fuse
- 11 Rated voltage/consumption
- 12 Load-bearing capacity

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Contents

1	General instructions	21	5.5.3	Mounting of height adapter at the rear	31
1.1	Introduction	21	5.5.4	Completing assembly	31
1.2	Proper use	21	6	Operation	31
1.3	Warranty	21	6.1	Control.	31
1.4	Repeat use/reconditioning	21	6.1.1	Lifting.	31
1.5	Disposal	21	6.1.2	Lowering	32
1.6	CE mark.	21	6.1.3	EMERGENCY-STOP	32
2	Safety instructions.	21	6.2	Using the bathlift	32
2.1	Warnings and symbols.	21	6.3	Transfer aids	33
2.2	General instructions.	22	6.4	Troubleshooting	34
2.3	Operating instructions	22	7	Maintenance and servicing	35
2.4	Instructions for maintenance and servicing	23	7.1	Seat and backrest	35
2.5	Instructions for recharging of battery	23	7.1.1	Cleaning and protection	35
3	Description	24	7.1.2	Cleaning	35
3.1	Scope of delivery	24	7.2	Charger and battery	35
3.2	Accessories	24	7.3	Maintenance	35
3.3	Technical data	25			
3.4	Summary description	25			
4	Transport	26			
5	Commissioning	26			
5.1	Recharging of battery.	26			
5.2	Mounting of suction cups	27			
5.2.1	Preparing assembly	27			
5.2.2	Mounting of suction cups at the front	27			
5.2.3	Mounting of suction cups at the rear.	27			
5.2.4	Completing assembly.	28			
5.3	Mounting bathlift in the bath.	28			
5.4	Removing bathlift from the bath.	29			
5.5	Mounting of height adapters (optional accessory).	30			
5.5.1	Preparing assembly	30			
5.5.2	Mounting of height adapter at the front	30			

1 General instructions

1.1 Introduction

This operating manual contains information and instructions regarding the safe and proper use of your **AQUATEC FORTUNA** bathlift.

By following these instructions, you prevent risk and enhance the service life of your **AQUATEC FORTUNA**.

In the text, we refer to graphics and individual items in figures. These references are shown in brackets.

Example: (3, Fig. 1) refers to figure 1, item 3.

Keep the documents supplied with your **AQUATEC FORTUNA** in a safe place for future reference.

1.2 Proper use

The **AQUATEC FORTUNA** is designed exclusively for the lowering and lifting of persons in a bathtub.

Any other use is forbidden.

Never use the **AQUATEC FORTUNA** as an aid to climb in or out of the bath, as a support, lifting platform or other similar purposes.

1.3 Warranty

Your **AQUATEC FORTUNA** comes with 3 year **AQUATEC** warranty.

Not covered by warranty are the covers, suction cups and the battery.

The warranty period begins at the date of delivery as shown on the **AQUATEC** docket.

1.4 Repeat use/reconditioning

The **AQUATEC FORTUNA** is designed for repeat use and can be easily moved from one bath to another. The maximum number of repeat uses depends on the general state and condition of the device.

For the reconditioning of the **AQUATEC FORTUNA** always comply with the relevant reworking and hygiene instructions of **AQUATEC**. These instructions are available on request from **AQUATEC**.

1.5 Disposal

Important!

The **AQUATEC FORTUNA** does not contain any hazardous substances.

To dispose of your **AQUATEC FORTUNA**, comply with the applicable statutory waste regulations. Certain parts of the device are identified as to their material and can be returned for recycling.

Damaged or spent batteries must be disposed of through the local recycling scheme.

1.6 CE mark

The **AQUATEC FORTUNA** complies with EU Directive 93/42/EEC for medical equipment.

2 Safety instructions

2.1 Warnings and symbols

The following warning symbols and signs are used in this manual:

*	CAUTION!
	This notice indicates a possible risk. If the relevant instructions are not adhered to, there is a risk of injury and damage to property.

Important!

This symbol is used to mark additional instructions that are not directly related to potential risks.

* In the text, the asterisk is replaced by a picture illustrating the type of risk.

2.2 General instructions

The **AQUATEC FORTUNA** may only be used for the lowering and lifting of persons in a bathtub.

If you have been advised for health reasons not to take a bath on your own, use the device only under supervision.

Example: You might be unable to reach the EMERGENCY-STOP button at the hand control device.

Never allow children to play with the **AQUATEC FORTUNA**.

Minors may only operate the **AQUATEC FORTUNA** under supervision by an adult.

Keep your **AQUATEC FORTUNA** clean and service it regularly according to the servicing instructions in this manual.

If you detect any malfunction or defect, immediately contact the vendor.

Do not make any modifications to the device.

2.3 Operating instructions

Prior to taking a bath, complete a test run* to ensure that the **AQUATEC FORTUNA** works properly.

Also check the bath temperature before you use the **AQUATEC FORTUNA**.

Observe the details given on the type plate. Do not overload the **AQUATEC FORTUNA**.

If you are using bath oils or bath salts, do not exceed the recommended dosage. Bath salts, etc. may impair the operation of the lifting scissors.

The hand control device may only be used, when the backrest is properly mounted and in upright position.

When transporting the device and when it is not used for a prolonged period of time, remove the battery from the slot. Do not use any other battery than that supplied with your **AQUATEC FORTUNA**.

Do not reach between the base plate and the seat plate.

* Move lift without load once to the upper stop and then to the lower stop.

2.4 Instructions for maintenance and servicing

When cleaning the **AQUATEC FORTUNA**, take precautions to prevent injury at the swivelling parts of the device and between the lifting scissors.

Never clean the backrest with a power hose.

For cleaning, use only solvent-free household detergents.

The temperature of the water may not exceed 60 °C.

All repairs to the **AQUATEC FORTUNA** must be completed by an authorised dealer.

2.5 Instructions for recharging of battery

Protect battery against fire and heat. Do not dispose of by burning.

Do not open the battery.

If the battery is damaged, diluted sulphuric acid might come into contact with skin and clothing. Immediately wash with plenty of water.

In the event of eye contact with diluted sulphuric acid, immediately rinse eye with plenty of water and consult a physician.

Do not touch the contacts of the battery with objects made in conducting materials (e.g. knitting needles). Do not immerse the battery in water.

Recharge the battery after each use of the device, using the supplied charger. Do not use any other recharging device.

If the device is not used for a prolonged period of time, recharge the battery once every month to prolong its service life.

To recharge the battery, move it to a dry room (do not recharge in the bathroom).

Dispose of spent batteries according to the statutory regulations.

Do not use the battery for any purpose other than the operation of the **AQUATEC FORTUNA**. Do not use the charger to recharge any other batteries than that supplied with your bathlift.

Insert the battery in the charger and connect the charger to the power mains.

3 Description

3.1 Scope of delivery

The **AQUATEC FORTUNA** is supplied with the following parts (figures 1 to 3):

Seat (figure 1) consisting of:

- 1 Sideflap
- 2 Seat plate
- 3 Hinge for backrest
- 4 Spindle holder
- 5 Lifting scissors
- 6 Sliding rail
- 7 Spindle foot support
- 8 Base plate

Suction cups (figure 1)

- 9 Front suction cup
(single-strap cup with head)
- 10 Rear suction cup
(double-strap screw-in cup)

Backrest (figure 2) consisting of:

- 1 Battery compartment
- 2 Electronic casing
- 3 Backrest plate
- 4 Spindle
- 5 Spindle foot
- 6 Spindle catch
- 7 Curled cable of hand control
- 8 Hand control
- 9 Hook for hand control

Battery and charger (figure 3)

- 2 Battery
- 5 Charger

Operating manual (no figure)

Cover (no figure)

Seat cover
Backrest cover

3.2 Accessories

The following accessories are available for your **AQUATEC FORTUNA**:

Height adapter (figure 12)

The height adapter (3, figure 12) is required for baths with a tub height of more than 42 cm. With the adapter, the height of the **AQUATEC FORTUNA** can be adjusted in steps of 2, 4 or 6 cm to ensure that the seat of the **AQUATEC FORTUNA** is level with the edge of the bath.

For installation instructions for the height adapter, please refer to chapter 5.5.

Pommel unit (stopper) (figure 16)

The pommel unit prevents the user from slipping from the seat. The pommel unit can be easily mounted on the seat plate.

The pommel unit is available in two sizes.

Sideflap diverter (figure 17)

The sideflap diverter is mounted onto the handrail of the bath. It serves as a guide rail and ensures that the seat is not blocked by the handrail while the **AQUATEC FORTUNA** is lifted.

Head support/special backrest (figure 18)

The head support (1, figure 18) supports the head and neck of the person using the bath-lift. In combination with the adjustable side sections of the special backrest (2, figure 18), it provides additional safety.

The head support must be secured to the backrest by means of the supplied mounting set.

Suction cup (no figure)

For bathes with anti-glide or dirt-repellent base, we provide special large-size suction cups.

3.3 Technical data

Bathlift

Seat length:	635 mm
Seat width with folded out sideflaps:	700 mm
Seat width with folded up sideflaps:	380 mm
Seat height at lowest position:	70 mm
Seat height at highest position:	420 mm
Seat depth from the lower edge of the backrest to the seat edge:	460 mm
Backrest height:	613 mm
Backrest width:	360 mm
Max. load capacity:	120 kg
Total weight without battery:	12.4 kg
Weight of seat:	7.9 kg
Weight of backrest:	4.5 kg
Weight of battery:	1.6 kg
Duration of lifting/lowering procedure (with load of 80 kg):	UP approx. 44 sec. DOWN approx. 34 sec.
Operating voltage:	12 V
Max. current consumption:	12 A
Rated input:	100 VA
Noise emission of drive without load (according to DIN 3746):	47 dB (A)
Medical device of type B	
Protection class:	IP 54
Operating factor:	5 %

Battery:	12 V/lead
Included fuse:	5 A (slow-blow)

Charger

Input:	230 V AC, 50/60 Hz, 55 mA/10.5 W
Input (USA and CDN):	120 V AC, 50/60 Hz, 55 mA/10.5 W
Output:	12 V DC, 400 mA/4.8 VA
Weight:	0.5 kg
Recharging period from low-voltage alarm:	approx. 5 h

The type plate of the **AQUATEC FORTUNA** (figure 15) shows the main device specifications.

It is mounted at the rear of the backrest.

3.4 Summary description

The main components of the **AQUATEC FORTUNA** are made from robust plastic materials. The sliding rails, the lifting scissors and the spindles are in stainless steel.

To facilitate handling, the backrest can be easily removed from the seat.

The seat and backrest are equipped with latex-free covers. Do not use the **AQUATEC FORTUNA** without covers at the seat and backrest.

The covers can be removed for cleaning.

The hand control is waterproof and floats on water. It contains all operating elements and a display showing the battery charge level.

The charger is only suitable for connection to the mains specified on the device. During recharging, an indicator lamp mounted on the charger is continuously on.

4 Transport

Important!

- When folded up, the backrest is not secured to the seat.
- Do not remove the suction cups when transporting the bathlift.
- Prior to moving the device, remove the battery from the compartment in the backrest.

Dismantle the **AQUATEC FORTUNA** into two parts (seat and backrest).

For detailed instructions on how to dismantle the device, please refer to chapter 5.4.

5 Commissioning

Important!

- Prior to commissioning, inspect all parts for damage caused by transport.
- Assemble the bathlift according to the instructions in this manual.

5.1 Recharging of battery

CAUTION!

Risk of injury from electric shock

Recharge the battery only in dry rooms (do not recharge in the bathroom).

Important!

- Recharge the battery each time you wish to use the bathlift.
 - Ensure that the current specifications shown on the charger correspond to those of the power mains.
1. Insert the plug (1, figure 3) of the charger (5, figure 3) into the socket (3, figure 3) of the battery (2, figure 3).
 2. Insert the power plug (4, figure 3) of the charger into the power socket.
- The red indicator lamp (6, figure 3) on the charger is on until the battery is fully recharged.
3. Disconnect the charger from the power supply.
 4. Disconnect the plug of the charger from the battery socket.

5.2 Mounting of suction cups

	CAUTION! Risk of injury from moving lifting scissors Do not reach between the lifting scissors during assembly.
---	---

Important!

To dismantle the suction cups, follow these instructions in reverse order.

5.2.1 Preparing assembly

1. Position the seat on its side.
2. Carefully pull the base plate (8, figure 1) from the seat plate (2, figure 1).

The seat is now laying on its side.

5.2.2 Mounting of suction cups at the front

Important!

- The procedure is the same for the right and the left suction cup.
- The front suction cups are equipped with one strap and a head that can be inserted into the plate.

1. Press the red suction cup lid (2, figure 4) from the base into the keyhole (3, figure 4) in the base plate.

Important!

Use a lubricant to facilitate the insertion.

2. Apply a thin layer of conventional hand cream onto the head of the suction cup (4, figure 4).

Important!

The strap with the toggle must face towards the back.

3. Insert the suction cup head into the larger bore of the keyhole.
4. Pull the suction cup head forward into the smaller opening of the keyhole.
5. Bend up the strap (5, figure 4) and insert the toggle into the groove (1, figure 4).
6. Push in the red suction cup lid.

5.2.3 Mounting of suction cups at the rear

Important!

- The procedure is the same for the right and the left suction cup.
- The rear suction cups are equipped with two straps and a threaded bolt.
- Install the cups in such a way that at least one strap of each cup can be reached when the lift is mounted into the bath.

Screw suction cup (1, figure 5) into the provided threaded liner (2, figure 5).

The threaded liner is located on the side of the base plate (3, figure 5), which also carries the spindle foot support (4, figure 5).

5.2.4 Completing assembly

1. Carefully push together the lifting scissors.
2. Position the seat on the mounted suction cups.

5.3 Mounting bathlift in the bath

	CAUTION!
	Sliding of seat if suction cups are not properly fixed • Prior to installing the lift in the bath, clean the bathtub and the suction cups. • After installation, do not move the AQUATEC FORTUNA in the bath.

Important!

- Prior to installation, check that all suction cups are properly mounted.
 - Ensure that the battery is fully recharged.
 - When lifting, please remember that the seat has a weight of approx. 8 kg, while the backrest weighs about 4.5 kg (without battery).
 - If your bath is equipped with a handle, install the side flap diverter (optional accessory).
1. Position the seat (2, figure 6) onto the base of the empty bath. The spindle foot support (3, figure 6) must thereby face the top end of the bath opposite the drain (1, figure 6).
 2. Fold sidefolds of the seat outwards (figure 7).
 3. Position the backrest (without battery) (1, figure 8) on the seat (7, figure 8), ensuring that the spindle foot (4, figure 8) faces the spindle foot support (6, figure 8).

4. Lift backrest slightly by holding it at the head end and place it onto the hinges (5, figure 8) on both side of the seat to the stop.
5. Press spindles (2, figure 8) into the spindle catch (3, figure 8) until they are engaged.

	CAUTION!
	Risk of damage if spindle foot becomes detached • Prior to folding up the backrest, check the spindle to ensure that it is properly engaged in the catch. • After the backrest is folded up, check spindle foot again for proper position and locking.

6. Carefully lift backrest by pulling it up at the head end (figure 8).

The spindle is thereby removed from the catch. The pegs (6, figure 9) slide along the spindle holder (2, figure 9) and engage again. The spindle foot (3, figure 9) is positioned in the spindle foot support (4, figure 9) base plate, and the spindle foot engages in the locking device (5, figure 9).

7. Insert the charged battery (2, figure 10) into compartment (1, figure 2) in the backrest.
8. Push in battery until it engages (figure 10).

The **AQUATEC FORTUNA** is now ready for operation.

5.4 Removing bathlift from the bath

1. Drain off all water from the bath.
2. Lower the **AQUATEC FORTUNA** by pressing the DOWN button (5, figure 13) (1, figure 11) until the backrest reaches the stop.
3. To remove the battery, hold it at the handle plate (1, figure 10) and pull it slightly upwards to disengage the locking mechanism. Then remove it from the battery compartment (2, figure 11).

Important!

When folding the backrest, ensure that the curled cable of the hand control is not damaged.

4. With one hand, hold the backrest at the head end.
5. With the other hand, press the grey locking device (5, figure 9) towards the base plate and hold it in this position.

The spindle foot is unlocked.

6. Fold the backrest slightly forward until the spindle foot (3, figure 9) is removed from the spindle foot support (4, figure 9).
7. Release the locking device.
8. With the free hand, hold the spindle (1, figure 9) at its foot.
9. Fold over backrest until it touches the seat (3, figure 11) while carefully pulling the spindle backwards and around the spindle holder (2, figure 9).
10. Press the spindle into the spindle catch (3, figure 8).

Important!

When lifting the bathlift components, please remember that the seat has a weight of approx. 8 kg, while the backrest weighs approx. 4.5 kg (without battery).

11. Lift backrest slightly by holding it at the head end.
12. Pull the backrest forward to disengage it from the hinges at the seat (4, figure 11).
13. Lift the backrest from the bath.
14. Fold sidefolds of the seat inwards (5, figure 11).

Important!

Release suction cups at the rear simultaneously!

15. To do this, pull the straps of the suction cups (10, figure 1).
16. Lift the rear end of the seat (6, figure 11).

The suction cups at the front are thereby released.

Important!

If your bathlift is equipped with height adapters (optional accessory), the suction cups at the front must be released by pulling the straps.

17. Lift the seat from the bath.
18. Store all components of the **AQUATEC FORTUNA** in a suitable room.

5.5 Mounting of height adapters (optional accessory)

	CAUTION!
	Risk of injury from moving lifting scissors Do not reach between the lifting scissors. Sliding of seat if suction cups are not properly fixed Do not use more than 3 height adapters per suction cup.

Important!

- To remove the height adapter, follow the instructions below in reverse order.
- If a second or third height adapter must be mounted on a suction cup, screw them into the thread of the already mounted adapter.

5.5.1 Preparing assembly

1. Position the seat on its side.
2. Carefully pull the base plate (8, figure 1) from the seat plate (2, figure 1).

The seat is now laying on its side.

5.5.2 Mounting of height adapter at the front

Important!

- The procedure is the same for the right and the left suction cup.
- When using height adapters, you cannot mount the original suction cups.

1. Press the red suction cup lid (2, figure 4) from the base into the keyhole (3, figure 4) in the base plate.
2. Ensure that the toggle at the strap (5, figure 4) remains outside the groove (1, figure 4).
3. Pull suction cup towards the back and remove it.
4. Press suction cup lid into the keyhole.

Important!

The peg at the base of the adapter insert must engage in the groove of the base plate.

5. Insert the adapter insert (1, figure 12) in to the open section of the bore (2, figure 12).
6. Screw the height adapter (3, figure 12) onto the thread of the adapter insert.
7. Screw the supplied replacement suction cups (double-strap cups) (4, figure 12) into the height adapter.

5.5.3 Mounting of height adapter at the rear

Important!

The procedure is the same for the right and the left suction cup.

1. Remove the suction cup from the base plate.
2. Screw the height adapter into the threaded liner in the base plate.
3. Screw the suction cup onto the height adapter.

5.5.4 Completing assembly

1. Carefully push together the lifting scissors.
2. Position the seat on the suction cups.

6 Operation

Important!

Do not fold the backrest if there is water in the bath.

6.1 Control

Important!

If the battery is empty, the red indicator light is on (2, figure 13). In this case, the lift cannot be lowered. Lifting is however possible. Immediately recharge the battery.

The **AQUATEC FORTUNA** is controlled by means of the hand control device (figure 13).

6.1.1 Lifting

Important!

- When the button for lifting/lowering is released, the lift is automatically stopped.
- If the backrest is folded down, it is moved to the upright position.

Press the UP button (3, figure 13) at the hand control (4, figure 13).

The backrest is moved to the upright position (1, figure 14), and the seat is lifted (2, figure 14).

6.1.2 Lowering

Important!

- When the button for lifting/lowering is released, the lift is automatically stopped.
- If the seat is already fully lowered, the backrest is folded backwards.

Press the DOWN button (5, figure 13) at the hand control (4, figure 13).

The seat is lowered (2, figure 14), and the backrest is folded backwards (1, figure 14).

6.1.3 EMERGENCY-STOP

Important!

Press the EMERGENCY-STOP button only in the case of an emergency.

Such a situation might occur when the UP or DOWN button is jammed. In such a case, press the EMERGENCY-STOP button (1, figure 13).

All movements of the **AQUATEC FORTUNA** are immediately stopped.

When the EMERGENCY-STOP button is released, the **AQUATEC FORTUNA** can again be operated normally, provided that no repairs are needed.

6.2 Using the bathlift

	CAUTION!
	Sliding of seat if suction cups are not properly fixed Prior to using the bathlift, ensure that it is securely positioned and cannot tilt. Also check the position of the AQUATEC FORTUNA in the bath and ensure that the spindle foot is properly inserted into the spindle foot support.

The **AQUATEC FORTUNA** must be installed in the bath as described in chapter 5.3.

1. Press the UP button (3, figure 13) to lift the seat to the height of the edge of the bath.

The sidefolds are folded out and laying on the bathtub rim.

2. Fill the bath with water and check the temperature.
3. Sit down on the sideflap nearest to you as if you would sit down on a chair.
4. Move yourself towards the centre of the seat.
5. Turn yourself so that you can be comfortably lowered into the water. Lift one leg after the other over the rim of the bathtub.
6. Press the DOWN button (5, figure 13) to lower yourself into the water.

As soon as the seat has reached its lowest position, you can lower the backrest so that you can comfortably lie in the bath.

7. To lower the backrest, keep the DOWN button pressed.

To lift yourself out of the bath, complete the above steps in reverse order.

To lift the seat, press the UP button (3, figure 13).

8. After a bath, remove the battery from the compartment and recharge it as described in chapter 5.1
or
dismantle and remove the **AQUATEC FORTUNA** from the bath as described in chapter 5.4.

6.3 Transfer aids

If you have difficulty to position yourself on the **AQUATEC FORTUNA**, you might consider using transfer aids.

Your medical supplies centre can advise you on suitable transfer aids.

AQUATEC DISK XL (figure 19)

The **AQUATEC DISK XL** is placed onto the seat of the **AQUATEC FORTUNA**. It is used to turn persons into the bath position.

AQUATEC TRANS XL (figure 20)

The **AQUATEC TRANS XL** is placed onto the seat of the **AQUATEC FORTUNA**. It is used to move and turn persons into the bath position.

AQUATEC STEP (figure 21)

The **AQUATEC STEP** is placed onto the bathroom floor in front of the bathtub. It facilitates mounting and dismantling, if the feet would otherwise not reach the floor.

6.4 Troubleshooting

Important!

The following table shows a number of common malfunctions and their probable cause. If a malfunction cannot be eliminated by following the instructions in the table, please contact your authorised dealer.

Problem	Possible causes	Remedy
No motor noise/lift does not move	Battery is not properly inserted Battery is empty Contacts are contaminated Contacts are bent Cable is damaged	Insert battery (see chapter 5.3) Recharge battery (see chapter 5.1) Clean contacts Readjust contacts Contact vendor for replacement of the cable
Lift can be moved upwards but not lowered	Low-voltage protection is triggered	Recharge battery
Lift stops during lifting operation	Load is excessive	The AQUATEC FORTUNA may only be used by persons weighing less than 120 kg.
Battery cannot be recharged	Fuse is defective Battery is defective	Contact your dealer to check the battery Replace battery
The bathlift is sliding in the bathtub	Suction cups are worn or damaged	Replace suction cups

7 Maintenance and servicing

7.1 Seat and backrest

	CAUTION! Risk of injury from moving lifting scissors When cleaning the seat, proceed with extreme care.
---	--

Important!

Hygiene is of the outmost priority when it comes to bathing equipment. Regularly clean both the seat and backrest and protect it from dust and dirt.

7.1.1 Cleaning and protection

Important!

Do not use abrasive cleaning agents.

1. To clean the **AQUATEC FORTUNA**, use conventional detergents and disinfectants. Clean the device with a soft cloth or brush.
2. Rinse the **AQUATEC FORTUNA** thoroughly with warm water.

Important!

Always dry the contacts in the battery compartment.

3. Dry the **AQUATEC FORTUNA** with a soft cloth.

7.1.2 Cleaning

1. Remove the covers from the seat and backrest.

	CAUTION! Risk of damage to covers Do not tumble-dry the covers.
---	--

2. The covers can be washed in the washing machine at max. 60 °C. Use a detergent for delicate fabrics.
3. Clean the bathlift as described in chapter 7.1.1.

4. Lubricate the sliding rail (6, figure 1) by applying a little Vaseline.

7.2 Charger and battery

	CAUTION! Risk of injury from electric shock • Prior to cleaning, disconnect the charger from the power supply. • Never touch the battery socket with an object made in a conducting material (e.g. knitting needles, metal pin). • To clean the charger and the battery, use only a dry cloth.
---	---

Clean the charger and the battery with a dry cloth.

7.3 Maintenance

The **AQUATEC FORTUNA** is maintenance-free, provided it is regularly cleaned as described above.

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© 2002 AQUATEC/DOLOMITE D-88316 Isny
(USA)**CLARKE HEALTH CARE PRODUCTS Inc.**