AQUATEC®

MEGAFLEX

SV BRUKSANVISNING

NO BRUKSANVISNING

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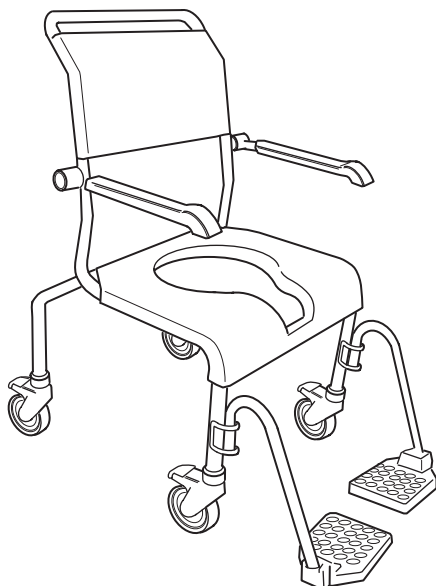
GB OPERATING INSTRUCTIONS

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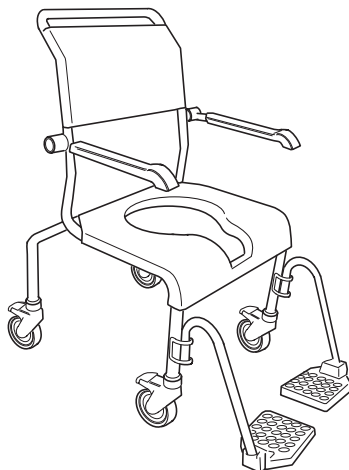
FR MANUEL D' UTILISATION

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SV BRUKSANVISNING

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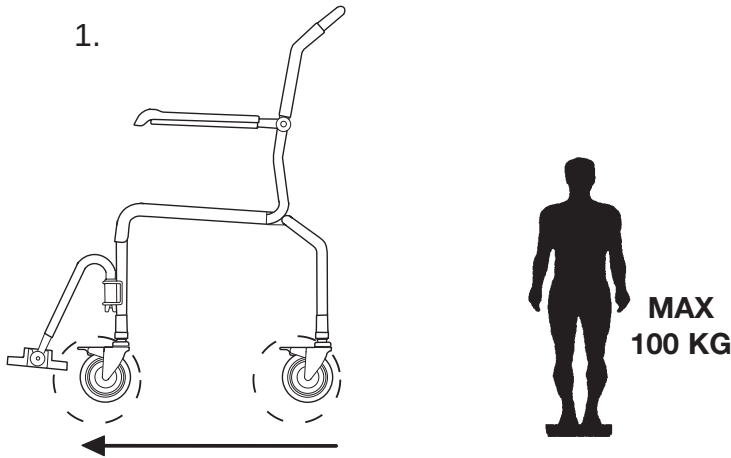
GB MAINTENANCE – SAFETY

MEGAFLEX Mobile Shower/Toilet chair is intended for use as a bedside toilet commode and/or as a shower chair. Do not stand on the foot supports, risk of tilting.

For optimum stability, position the link wheels forwards as shown in the fig. no 1. If the user has a tendency to fall forwards, also use the support or security belt. Not for use in pool or similar places where equipment is immersed in water. Minimum two wheels should be locked in position while toileting or showering. MEGAFLEX Mobile Shower chair is only for use on flat surfaces. To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the hygiene seat with solvent-free household cleaner. Keep the wheels clean. Avoid detergent based on lemon
- **Ensure that screws and adjustments** are properly tightened.
- Lubricate the wheels regularly with grease.
- **Immediately contact the nearest dealer if a fault should appear. Never use a defective chair.** Do not attempt repairs yourself! **DOLOMITE** will not accept responsibility for adaptations, faulty repairs, etc, that are outside our control.

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DOLOMITE AB
AQUATEC GmbH & Co.KG
MOBITEC Mobilitätshilfen GmbH