AQUATEC®

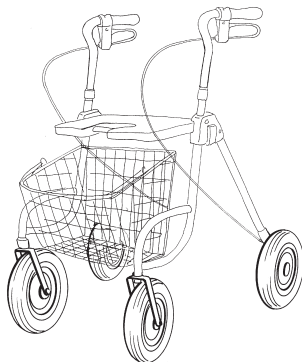
DOLOMITE SHOPPY

SV BRUKSANVISNING

GB OPERATING INSTRUCTIONS

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SHOPPY

SV BRUKSANVISNING

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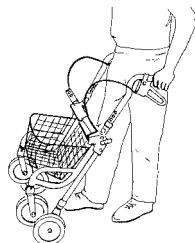
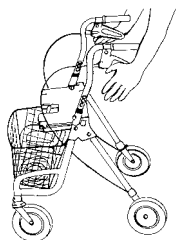
GB OPERATING INSTRUCTIONS

Study the operating instructions carefully **before** use.

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GB FOLDING

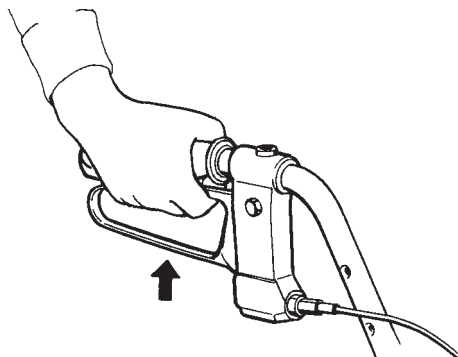
1. Lift up the red safety latch on left side of seat. Ensure it is secured to velcro pad.
2. Press down parking brakes.
3. Hold both handles.
4. Tilt walker backwards with back wheels staying on the floor, then gently pull the handles upwards, towards yourself for easy folding.

Lift handle to fold, lift the safety latch. Folded position. Suitable for carrying in stairs. Lift up the latch, so it attach to the Velcro. NB. Push the latch back to the locking position when using the product.

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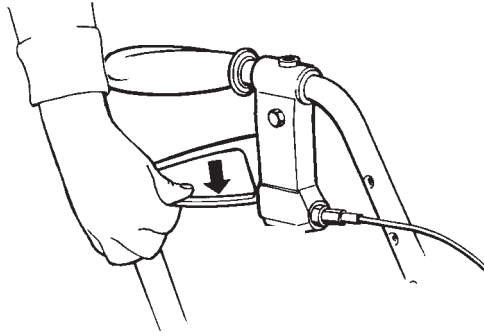
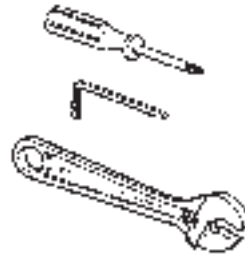
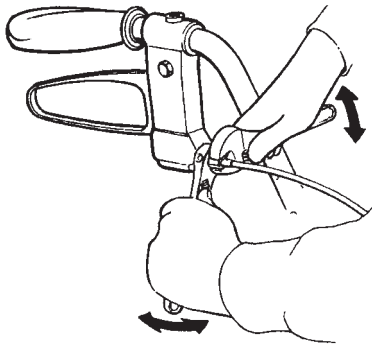
GB BRAKE

Press down carefully for parking brake.

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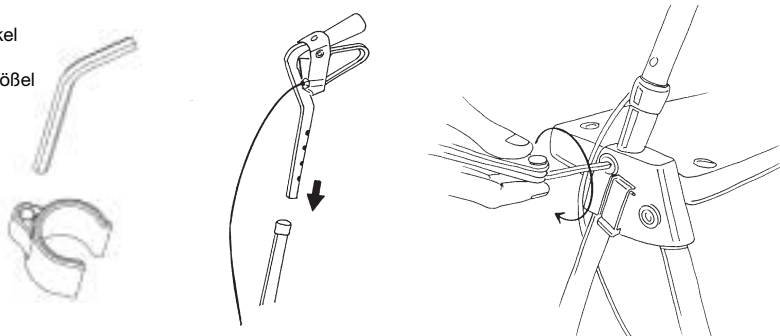
GB BRAKE

For desirable braking strenght, adjust
the brake wire. Suitable tools not
included.

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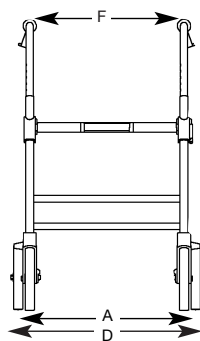
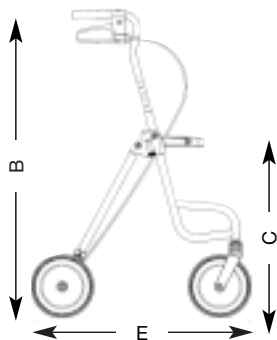
GB HANDLEBAR

1. First, loosen the screws from one side only, as you unscrew them with the Allen key. Remove the screws and the nuts which have been loosened on the other side.
 2. Place the handle tube into the frame tube hole provided for this purpose.
 3. Determine the height of the handle by means of the inserted handle tube.
 4. Place the nut in the hexagonal hole and feed the screw through the holes into the cast part, the frame tube and into the handle tube. Make sure that the holes are positioned on top of one another. Now fix the handle tube. At this point, tighten the screws firmly using the Allen key. After one side is fixed, proceed in the same way with the other side.
- Firstly adjust one handle and then the other. Not both at the same time.

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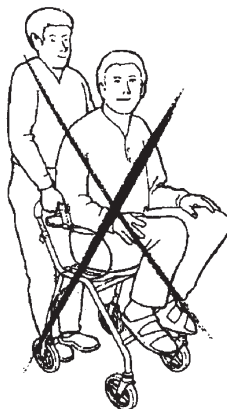
GB Not to be used for standing on or as a wheelchair.

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GB MAINTENANCE – SAFETY

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with solvent-free household cleaner. Keep the wheels clean.
- **Ensure that screws and adjustments are properly tightened.**
- Lubricate the wheels regularly with grease.
- **Immediately contact the nearest dealer if a fault should appear.**

Never use a defective walker. Do not attempt repairs yourself!

N.B. Your walker should only be used as a walking aid! User's max. weight: 275lbs. (125 kg, 20 stones). Dolomite will not accept responsibility for adaptations, faulty repairs, etc. that are outside their control.

The walker described complies to the regulations of the Medical Device Directive 93/42 EWG

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DOLOMITE AB
AQUATEC GmbH & Co.KG
AQUATEC Schmidt & Lenhardt GmbH